

Dia-REF**CLEANING REAGENT**

Cat. No.: 11500000 500 ml

PRODUCT NAME

Dia-REF

INTENDED USE**((For In Vitro Diagnostic Use Only))**

For use with ACL Coagulometer series (100-7000) instruments for adjusting and washing the instruments.

SUMMARY AND PRINCIPLE

The ACL equipments determine the endpoint of the clotting process by nephelometric method. The equipments need an emulsion for adjusting the calibration and for washing the fluid circuit between the working steps.

For any other information concerning the measurements see the corresponding manual of the equipments.

ACTIVE INGREDIENTS

Silicon emulsion	< 1,0 %
Sodium chloride	< 0,9 %
Sodium azide	< 0,01 %

PRECAUTIONS

- This product is ready for use.
- Do not use the reagent beyond the expiration date printed on the label.
- All reagents, waste and utilized disposable laboratory equipments should be considered as hazardous waste. Their handling and disposal should be done according to the valid hazardous material processing regulation.

INSTRUCTION FOR USE

This product is intended for educated laboratory personal.

The product has to be at room temperature well before use (24h)

Open the bottle and connect the suction tube to it, and fix the stopper again.

Prepare every reagents and disposables for the measurements according to the manual.

Do not mix this emulsion with the products of other manufacturer ! It may lead to erroneous result !

STORAGE AND STABILITY

The reagent is stable until the expiry date printed on the label, provided it is stored between 15-30 °C. If the reagent shows any visible turbidity, change of colour etc., it has to be changed !

Do not use the product after being frozen!

EXPECTED RESULTS

It's usage together with the reagents according to the manual leads to correct tests results.

LIMITATIONS

The reagent has to be used within the ambient temperature range of 15–35°C. Use of the reagent does not substitute the proper operation and maintenance of the instrument.

MATERIALS REQUIRED BUT NOT PROVIDED

For the availability of the reagents below please contact your local distributor.

- Prothrombin time (Dia-PT; Cat. No.: 81100, 81050)
- Activated partial thromboplastin time (Dia-PTT; Cat. No.: 71048)
- Fibrinogen (Dia-FIB; Cat. No.: 61060)
- Thrombin time test (Dia-TT; Cat. No.: 51036)
- IL reagent

MANUFACTURER 

DIAGON Ltd.

Baross u. 52. 1047 Budapest, Hungary

Tel.: +36 1 369 6500,

Fax: +36 1 369 6301

E-mail: diagon@diagon.comWeb: www.diagon.com