

# MAGLUMI IgE (CLIA)



130208001M



100



**Shenzhen New Industries  
Biomedical Engineering Co., Ltd**  
4F, Wearnes Tech Bldg,  
Science & Industry Park,  
Nanshan, Shenzhen, 518057 CHINA  
Tel. + 86-755-86028224  
Fax. + 86-755-26654850



**Lotus Global Co., Ltd**  
15 Alexandra Road  
London  
NW8 0DP  
UK  
Tel. + 44-20-75868010  
Fax. + 44-20-79006187



## FOR PROFESSIONAL USE ONLY

Store at 2...8 °C



COMPLETELY READ THE INSTRUCTIONS BEFORE  
PROCEEDING



## SYMBOLS EXPLANATIONS



Authorized Representative in Europe




Manufacturer



Attention. See Instructions For Use



Contents of kit

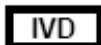

In vitro diagnostic medical device  
(In vitro diagnostic use)




Lot number



Catalogue Code



Expiry date (Use by...)

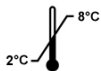

Temperature limitation  
(store at 2...8 °C)



Number of tests



Keep away from sunlight



Keep upright

## INTENDED USE

The kit has been designed for the quantitative determination of Immunoglobulin E (IgE) in human serum.

The method can be used for samples over the range of 0-3200 IU/ml.

The test has to be performed on the MAGLUMI chemiluminescence immunoassay (CLIA) fully auto analyzer (Including MAGLUMI 1000, MAGLUMI 2000, MAGLUMI 2000 Plus and new developed models).

## SUMMARY AND EXPLANATION OF THE TEST

IgE is one of the body's 5 classes (isotypes) of immunoglobulins (antibodies). Like other immunoglobulins, IgE is produced by B cells and plasma cells. In contrast to other immunoglobulins, the concentration of IgE in the circulation is very low. Immunoglobulin E in cord blood usually measures less than 1 U/ml (1 U = 2.4 ng). Generally, adult IgE levels are achieved by 5 to 7 years of age. Between the ages of 10 and 14 years, IgE levels may be higher than those in adults. After age 70 years, IgE levels may decline slightly and be lower than the levels observed in adults younger than 40 years.

Circulating IgE concentrations are very low because mast cells have a very high affinity for IgE (1010 mol/L<sup>-1</sup>) via their -heavy-chain Fc receptors (FcER). The synthetic rate for IgE is also very low. Immunoglobulin E attaches to mast cells and to basophils and activated eosinophils. Immunoglobulin E on mast cells has a half-life (T<sub>1/2</sub>) of more than 10 days.

In contrast to other immunoglobulins that bind to immunoglobulin Fc receptors only when antigen has been bound by an antibody, IgE will bind to FcER in the absence of antibody. Immunoglobulin E binding to mast cells "sensitizes" the mast cells to degranulate when multivalent antigens cross-link FcER-bound IgE. Intrinsically innocuous antigens that produce hypersensitivity responses are termed allergens.

## PRINCIPLE OF THE TEST

Sandwich immunoluminometric assay;

Use an anti-IgE monoclonal antibody to label ABEI, and use another monoclonal antibody to label FITC. Prediluted Sample, Calibrator or Control with ABEI Label, FITC Label and magnetic microbeads coated with anti-FITC are mixed thoroughly and incubated at 37

°C, forming a sandwich complex. In a magnetic field, decant the supernatant, then cycle washing for 1 time. Subsequently, the starter reagents are added and a flash chemiluminescent reaction is initiated. The light signal is measured by a photomultiplier as RLU within 3 seconds and is proportional to the concentration of IgE present in controls or samples.



## KIT COMPONENTS

### Material Supplies

Reagent Integral for 100 determinations	
<b>Nano magnetic microbeads:</b> TRIS buffer, 1.2%(W/V), 0.2%NaN <sub>3</sub> , coated with sheep anti- FITC polyclonal antibody.	2.5ml
<b>Calibrator Low :</b> bovine serum, 0.2%NaN <sub>3</sub> .	2.5ml
<b>Calibrator High :</b> bovine serum, 0.2%NaN <sub>3</sub> .	2.5ml
<b>FITC Label:</b> anti-IgE monoclonal antibody labeled FITC, contains BSA, 0.2%NaN <sub>3</sub> .	6.5ml
<b>ABEI Label:</b> anti-IgE monoclonal antibody labeled ABEI, contains BSA, 0.2%NaN <sub>3</sub> .	6.5ml
<b>Diluent:</b> 0.9% NaCl.	25.0ml
All reagents are provided ready-to-use.	

Reagent Vials in kit box	
Internal Quality Control: containing BSA, 0.2%NaN <sub>3</sub> . (target value refer to Quality Control Information date sheet)	2.0ml

#### Accessories Required But Not Provided

MAGLUMI Reaction Module	REF: 630003
MAGLUMI Starter 1+2	REF: 130299004M
MAGLUMI Wash Concentrate	REF: 130299005M
MAGLUMI Light Check	REF: 130299006M



#### Preparation of the Reagent Integral

Before the sealing is removed, gentle and careful horizontal shaking of the Reagent Integral is essential (avoid foam formation!) Remove the sealing and turn the small wheel of the magnetic microbeads compartment to and fro, until the colour of the suspension has changed into brown. Place the Integral into the reagent area and let it stand there for 30 min. During this time, the magnetic microbeads are automatically agitated and completely resuspended.

**Do not interchange integral component from different reagents or lots!**

#### Storage and Stability

- Sealed: Stored at 2-8 °C until the expiry date.
- Opened: Stable for 4 weeks. To ensure the best kit performance, it is recommended to place opened kits in the refrigerator if it's not going to be used on board during the next 12 hours.

- Keep upright for storage.

- Keep away from direct sunlight.

## CALIBRATION AND TRACEABILITY

### 1) Traceability

To perform an accurate calibration, we have provided the test calibrators standardized against the WHO Reference Reagent Serum Immunoglobulin E (IgE) Human.

### 2) 2-Point Recalibration

Via the measurement of calibrators, the predefined master curve is adjusted (recalibrated) to a new, instrument-specific measurement level with each calibration.

### 3) Frequency of Recalibration

- After each exchange of lot (Reagent Integral or Starter Reagents).
- Every 4 weeks and/or each time a new Integral is used (recommendation).
- After each servicing of the MAGLUMI Fully Auto analyzer.
- If controls are beyond the expected range.

## SPECIMEN COLLECTION AND PREPARATION

Sample material: serum

Collect samples using standard procedures.

Store at 2-8 °C below -20 °C

Avoid repeated freezing and thawing cycles, stored samples should be thoroughly mixed prior to use (Vortex mixer).

Please ask local representative of SNIBE for more details if you have any doubt.

#### Vacuum Tubes

041120113-v1.0-EN

(a) Blank tubes are recommended type for collecting samples.

(b) Please ask SNIBE for advice if special additive must be used in sample collecting.

#### Specimen Conditions

- Do not use specimens with the following conditions:
  - heat-inactivated specimens;
  - Cadaver specimens or body fluids other than human serum;
  - Obvious microbial contamination.
- Use caution when handling patient specimens to prevent cross contamination. Use of disposable pipettes or pipette tips is recommended.
- Inspect all samples for bubbles. Remove bubbles with an applicator stick prior to analysis. Use a new applicator stick for each sample to prevent cross contamination.
- Serum specimens should be free of fibrin, red blood cells or other particulate matter.
- Ensure that complete clot formation in serum specimens has taken place prior to centrifugation. Some specimens, especially those from patients receiving anticoagulant or thrombolytic therapy, may exhibit increased clotting time. If the specimen is centrifuged before a complete clot forms, the presence of fibrin may cause erroneous results.

#### Preparation for Analysis

- Patient specimens with a cloudy or turbid appearance must be centrifuged prior to testing. Following centrifugation, avoid the lipid layer (if present) when pipetting the specimen into a sample cup or secondary tube.
- Specimens must be mixed **thoroughly** after thawing by **low** speed vortexing or by gently inverting, and centrifuged prior to use to remove red blood cells or particulate matter to ensure consistency in the results. Multiple freeze-thaw cycles of specimens should be avoided.
- All samples (patient specimens or controls) should be tested within 3 hours of being placed on board the MAGLUMI System. Refer to the SNIBE service for a more detailed discussion of onboard sample storage constraints.

#### Storage

- If testing will be delayed for more than 8 hours, remove serum or plasma from the serum or plasma separator, red blood cells or clot. Specimens removed from the separator gel, cells or clot may be stored up to 24 hours at 2-8°C.
- Specimens can be stored up to 30 days frozen at -20°C or colder.

#### Shipping

- Before shipping specimens, it is recommended that specimens be removed from the serum or plasma separator, red blood cells or clot. When shipped, specimens must be packaged and labeled in compliance with applicable state, federal and international regulations covering the transport of clinical specimens and infectious substances. Specimens must be shipped frozen (dry ice). Do not exceed the storage time limitations identified in this section of the package insert.

## WARNING AND PRECAUTIONS FOR USERS



- For use in *IN-VITRO* diagnostic procedures only. Package insert instructions must be carefully followed. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this package insert.

#### Safety Precautions

- CAUTION:** This product requires the handling of human specimens.
- The calibrators in this kit are prepared from bovine serum

products. However, because no test method can offer complete assurance that HIV, Hepatitis B Virus or other infectious agents are absent; these reagents should be considered a potential biohazard and handled with the same precautions as applied to any serum or plasma specimen.

- All samples, biological reagents and materials used in the assay must be considered potentially able to transmit infectious agents. They should therefore be disposed of in accordance with the prevailing regulations and guidelines of the agencies holding jurisdiction over the laboratory, and the regulations of each country. Disposable materials must be incinerated; liquid waste must be decontaminated with sodium hypochlorite at a final concentration of 5% for at least half an hour. Any materials to be reused must be autoclaved using an overkill approach (USP24, 2000, p.2143). A minimum of one hour at 121 °C is usually considered adequate, though the users must check the effectiveness of their decontamination cycle by initially validating it and routinely using biological indicators.
- It is recommended that all human sourced materials be considered potentially infectious and handled in accordance with the OSHA Standard on Bloodborne Pathogens 13. Biosafety Level 214 or other appropriate biosafety practices should be used for materials that contain or are suspected of containing infectious agents.
- This product contains Sodium Azide; This material and its container must be disposed of in a safe way.
- Safety data sheets are available on request.

#### Handling Precautions

- Do not use reagent kits beyond the expiration date.
- Do not mix reagents from different reagent kits.
- Prior to loading the Reagent Kit on the system for the first time, the microbeads requires mixing to re-suspend microbeads that have settled during shipment.
- For microbeads mixing instructions, refer to the KIT COMPONENTS, Preparation of the Reagent Integral section of this package insert.
- To avoid contamination, wear clean gloves when operating with a reagent kit and sample.
- Over time, residual liquids may dry on the kit surface, please pay attention the silicon film still exists on the surface of the kit.
- For a detailed discussion of handling precautions during system operation, refer to the SNIBE service information.

#### TEST PROCEDURE

To ensure proper test performance, strictly adhere to the operating instructions of the MAGLUMI Fully Auto analyzer. Each test parameter is identified via a RFID tag on the Reagent Integral. For further information please refer to the MAGLUMI Chemiluminescence Analyzer Operating Instructions.

Auto dilution 1:10 20µl +180µl	Sample diluent
20µl +40µl +40µl +20µl	Sample, calibrator or controls ABEI Label FITC Label Nano magnetic microbeads
15 min	Incubation
400µl	Cycle washing
3 s	Measurement

#### DILUTION

Samples with concentrations above the measuring range can be diluted. After manual dilution, multiply the result by the dilution factor. After dilution by the analyzers, the analyzer software automatically takes the dilution into account when calculating the sample concentration.

Availability of sample dilution by analyzer please refers to the MAGLUMI analyzer user software program. Dilution settings

please follow MALGUMI analyzer operating instructions.

#### QUALITY CONTROL

- Observe quality control guidelines for medical laboratories
- Use suitable controls for in-house quality control. Controls should be run at least once every 24 hours when the test is in use, once per reagent kit and after every calibration. The control intervals should be adapted to each laboratory's individual requirements. Values obtained should fall within the defined ranges. Each laboratory should establish guidelines for corrective measures to be taken if values fall outside the range.

#### LIMITATIONS OF THE PROCEDURE

##### 1) Limitations

Patients with malignancies may exhibit IgE values within the normal range. IgE concentrations may be elevated in case of liver cirrhosis, hepatitis or tyrosinaemia. Thus, IgE determination is more suitable for therapeutic monitoring and follow-up as well as for a comparison with histological results. IgE serum levels may only be interpreted in context with the clinical picture and other diagnostic procedures. The IgE assay should not be used as the only criterion for cancer screening.

##### 2) Interfering Substances

No interference with test results is seen by concentrations of bilirubin<0.06mg/ml, haemoglobin<16mg/dl or triglycerides<12.5mg/ml.

##### 3) HAMA

Patient samples containing human anti-mouse antibodies (HAMA) may give falsely elevated or decreased values. Although HAMA-neutralizing agents are added, extremely high HAMA serum concentrations may occasionally influence results.

##### 4) High-Dose Hook

No high-dose hook effect was seen for IgE concentrations up to 10,000 IU/ml.

#### RESULTS

##### 1) Calculation of Results

The analyzer automatically calculates the IgE concentration in each sample by means of a calibration curve which is generated by a 2-point calibration master curve procedure. The results are expressed in IU/ml. For further information please refer to the MAGLUMI Chemiluminescence Analyzer Operating Instructions.

##### 2) Interpretation of Results

- Results of study in clinical centers with group of individuals, 95% of the results were:<190IU/ml.
- Results may differ between laboratories due to variations in population and test method. If necessary, each laboratory should establish its own reference range.

#### PERFORMANCE CHARACTERISTICS

##### 1) Precision

Intra-assay coefficient of variation was evaluated on 3 different levels of control serum repeatedly measured 20 times in the same run, calculating the coefficient of variation.

##### Intra-assay precision

Control	Mean(IU/ml)	SD(IU/ml)	CV%
Level 1	51.36	1.97	1.89
Level 2	112.04	2.16	1.93
Level 3	159.83	3.92	2.46

Inter-assay coefficient of variation was evaluated on three batches of kits. Repeatedly measured 3 different levels of control serum 21 times, calculating the coefficient of variation.

##### Inter-assay precision

Control	Mean(IU/ml)	SD(IU/ml)	CV%
---------	-------------	-----------	-----

Level 1	52.45	3.56	6.79
Level 2	114.92	5.76	5.01
Level 3	162.63	8.81	5.42

## 2) Analytical Sensitivity

The sensitivity is defined as the concentration of IgE equivalent to the mean RLU of 20 replicates of the zero standard plus two standard deviations corresponding to the concentration from the standard curve. The sensitivity is typically less than 1.25IU/ml.

## 3) Specificity

The specificity of the IgE assay system was assessed by measuring the apparent response of the assay to various potentially cross reactive analytes.

Compound	Concentration	Cross reactivity
IgG	100 µg/ml	0.7%

## 4) Recovery

Consider calibrator high of known concentration as a sample, dilute it by 1:2 ratio with diluents, and measure its diluted concentration for 10 times. Then calculate the recovery of measured concentration and expected concentration. The recovery should be within 90% -110%.

Expected	Mean Measuring	Recovery
87.241 IU/ml	83.794 IU/ml	96%

## 5) Linearity

Use IgE calibrator to prepare the six-point standard curve, measuring all points' RLU except point A, and then do four-parameter linear fitting in double logarithm coordinate, the absolute linear correlation coefficient(r) should be bigger than 0.9800.

Calibrator Point	Concentration IU/ml	Absolute linear correlation coefficient (r)
A	0	
B	5	r=0.9835
C	20	
D	40	
E	80	
F	320	

## 6) Method comparison

A comparison of MAGLUMI IgE (y) with a commercially available IgE test (x) using clinical samples gave the following correlations (IU/ml):

Linear regression

$$y = 1.16x - 114$$

$$r = 0.961$$

$$Sy.x = 195$$

Number of samples measured: 185

The sample concentrations were between 4 and 280 IU/ml.

## REFERENCES

- Pagana, K. D. & Pagana, T. J. (© 2007). Mosby's Diagnostic and Laboratory Test Reference 8th Edition: Mosby, Inc., Saint Louis, MO. Pp 564-566.
- Wu, A. (© 2006). Tietz Clinical Guide to Laboratory Tests, 4th Edition: Saunders Elsevier, St. Louis, MO. Pp 598-611.
- Kasper DL, Braunwald E, Fauci AS, Hauser SL, Longo DL, Jameson JL eds (2005). Harrison's Principles of Internal Medicine, 16th Edition, McGraw Hill, Pp 1921-1922, 1940-1945.
- Sanford KW, Roseff SD. Immunodeficiency Disorders. Henry's Clinical Diagnosis and Management by Laboratory Methods, 21st ed. RA McPherson and MR Pincus, eds. Saunders:New York, Pp 906-915, 2007.

- Tietz Textbook of Clinical Chemistry and Molecular Diagnostics. Burtis CA, Ashwood ER, Bruns DE, eds. St. Louis: Elsevier Saunders; 2006 Pp 571-573, 578, 580.
- Robert G. Hamilton, P. Brock Williams, Human IgE antibody serology: A primer for the practicing North American allergist/immunologist Specific IgE Testing Task Force of the American Academy of Allergy, Asthma & Immunology and the American College of Allergy, Asthma and Immunology, Journal of Allergy and Clinical Immunology, Volume 126, Issue 1, July 2010, Pages 33-38.
- Al-Herz W, McGeedy SJ, Gripp KW. 22q11.2 deletion syndrome and selective IgE deficiency: an association of a common chromosomal abnormality with a rare immunodeficiency. Am J Med Genet. 2004; 127A:99-100.
- De la Concha EG, Garcia-Rodriguez MC, Zabay JM. Functional assessment of T and B lymphocytes in patients with selective IgE deficiency. Clin Exp Immunol. Sep 1982; 49(3):670-6.
- Faulk WP, Kiyasu WS, Cooper MD. Deficiency of IgE. Pediatrics. Feb 1971; 47(2):399-404.
- Guill MF, Brown DA, Ochs HD. IgE deficiency: clinical spectrum and immunologic assessment. Ann Allergy. Jun 1989; 62(6):547-52.
- Homburger HA. The Laboratory Evaluation of Allergic Diseases: Part I: Measurement Methods for IgE Protein. Lab med 1991; 22:780 - 2.
- Johansson SGO. In vitro Diagnosis of Reagin-Mediated Allergic Diseases. Allergy 1978; 33:292 - 98.
- Burrows B, Martinez FD, Halonen M, Barbee RA, Cline MG. Association of Asthma with Serum IgE Levels and Skin-Test Reactivity to allergens. N Engl J Med 1989; 320:271 - 277.
- Loeffler J, Cawley LP, Moeder M. Serum IgE levels: Correlation with skin test sensitivity. Ann Allergy 1973; 31: 331-6.
- Greenberger P, Patterson R. Allergic bronchopulmonary aspergillosis and the evaluation of the patient with asthma. J All Clin Immunol 1988; 81(4):646 - 650.
- R. Patterson, et al. Serum Immunoglobulin E in pulmonary allergic aspergillosis. J All Clin Immunol 1972; 49(2):98- 99.
- Waldman TA, Polmar SH, Ballestra ST, Jost MC, Bruce RM, Terry WD. Immunoglobulin E in immunologic de-ficiency diseases. II. Serum IgE Concentration of Patients with Acquired Hypogammaglobulinemia, Thymoma and Hypogammaglobulinemia, Myotonic Dystrophy, Intestinal Lymphangiectasia and Wiskott-Aldrich Syndrome. J Immunol 1972; 109(2):304 - 10.
- Geha R, Reinherz E, Leung D, McKee KT, Schlossman S, Rosen FS. Deficiency of Suppressor T Cells in the Hyperimmunoglobulin E Syndrome. J Clin Invest 1981; 68:783 - 791.
- Kojima S, Yokogawa M, Tada T. Raised Levels of Serum IgE in Human Helminthiasis. Am J Trop Med Hyg 1972; 21(6):913 - 17.
- Guder WG, Narayanan S, Wisser H, Zawta B. List of Analytes Preanalytical Variables. Brochure in: Samples: From the Patient to the Laboratory. GIT-Verlag, Darmstadt 1996:16 - 17. ISBN 3-928865-22-6.