CORMAY IMMUNO- MULTICAL

(PL)

CORMAY IMMUNO- MULTICAL

Nr kat. 4-287 5 x 1 ml

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LITERATURA

1. Johnson, A.M.: A new international preparation for proteins in human serum, Arch. Pathol. Lab. Med. 117 (1993) 29-31.

Data wydania: 10. 2020.

CORMAY IMMUNO- MULTICAL

(EN)

CORMAY IMMUNO- MULTICAL

Cat. No 4-287 5 x 1 ml

INTENDED USE

CORMAY IMMUNO-MULTICAL is designed for use as multianalyte calibrator when measuring various serum proteins in human serum. Use in accordance with instructions enclosed to reagents for the performance of particular proteins.

The reagents must be used only for *in vitro* diagnostic, by suitably qualified laboratory personnel, only for the intended purpose, under appropriate laboratory conditions.

CHARACTERISTICS

The CORMAY IMMUNO-MULTICAL is a human biological fluid diluted in HEPES (pH 7.4) buffer containing stabilisers, sodium azide (< 1 g/l) and the following human proteins: alpha-1 antitrypsin, alpha-1 glycoprotein acid, antithrombin III, complement C3, complement C4, ceruloplasmin, haptoglobin, IgA, IgG, IgM and transferrin.

PREPARATION AND STORAGE

Calibrators are ready for use.

Calibrators stored at 2-8°C are stable until expiry date printed on the package. Store closed and avoid contamination.

NOTES

1. Calibrators must be used only for the purpose intended by suitably qualified laboratory personnel, under appropriate laboratory conditions.
2. This calibrator has been tested for the HIV antibody, HBsAg and HCV and found to be non-reactive. However this material should be handled as though capable of transmitting infectious disease.
3. Calibrators contain sodium azide (< 1 g/l) as a preservative. Avoid contact with skin and mucous membranes.

ASSIGNED VALUES

Assigned values for particular proteins vary from lot to lot are given in values sheet.

TRACEABILITY

Values assigned from the reference material ERM DA470k/IFCC.

WASTE MANAGEMENT

Please refer to local legal requirements.

LITERATURE

1. Johnson, A.M.: A new international preparation for proteins in human serum, Arch. Pathol. Lab. Med. 117 (1993) 29-31.

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