CORMAY ETHANOL CALIBRATORS

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INTENDED USE

Calibrators for determination of ethyl alcohol concentration. CORMAY ETHANOL CALIBRATORS kit is intended for calibration of ethanol determinations on automatic analysers. CORMAY ETHANOL CALIBRATOR 100 kit is intended for calibration of ethanol determinations manually performed. The calibrator must be used only for *in vitro* diagnostic, by suitably qualified laboratory personnel, only for the intended purpose, under appropriate laboratory conditions.

INTRODUCTION

Ethanol after consumption is absorbed in alimentary tract and transported with blood to whole organism. A little amount of ethanol is excreted with urine or exhaled with air, but majority is metabolized by liver to acetic aldehyde, then to acetate, finally to carbon dioxide and water. Ethanol consumption is often cause of various types of accidents, and ethanol poisoning may be cause of decease.

PACKAGE

CORMAY ETHANOL CALIBRATORS

	size	concentration
ETHANOL CALIBRATOR 1	5 x 1 ml	100 mg/dl
ETHANOL CALIBRATOR 2	5 x 1 ml	300 mg/dl

CORMAY ETHANOL CALIBRATOR 100

	size	concentration
ETHANOL CALIBRATOR 100	10 x 1 ml	100 mg/dl

The concentrations of ethyl alcohol are printed on the label of the each vial and in this kit insert.

STABILITY AND STORAGE

- Unopened calibrators are stable at 15-25°C until the expiration date printed on the package.
- Stability after opening:
The content of the ampoule must be used immediately after opening since concentration changes due to ethanol evaporation.
Opened ampoule can be used only once!
Do not freeze the standards!

PROCEDURE

- Calibrators are ready to use.
- Mix calibrators gently and accurately before use.

NOTES

- Do not use calibrators after expiry date.