## GLUCOSE STANDARD

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### INTENDED USE

GLUCOSE STANDARD 100 and GLUCOSE STANDARD 300 are dedicated for calibration of glucose determination in blood serum or plasma and cerebrospinal fluid, using manual assay.

The standards must be used only for *in vitro* diagnostic, by suitably qualified laboratory personnel, only for the intended purpose, under appropriate laboratory conditions.

### INTRODUCTION

Glucose is a simple six-carbon sugar. Oxidative metabolism of glucose provides the energy for most cellular processes. Glucose level in the blood is tightly controlled by several hormones. Elevated glucose level is the classic sign of diabetes mellitus. Glucose level abnormalities (hyper- or hypoglycemia) might be caused also by pancreas tumors and diseases of liver, thyroid gland or adrenal glands.

### COMPOSITION

GLUCOSE STANDARD 100 and GLUCOSE STANDARD 300 are single level liquid standards prepared in a buffer base.

The exact standard concentration is printed on the label of each vial.

### PREPARATION AND STORAGE

The standards are ready for use.

Standards when stored at 2-25°C are stable up to expiry date printed on the vial.

### WARNINGS AND NOTES

- Store standards tightly capped when not in use.
- Please refer to the MSDS for detailed information concerning safe storage and use of the product.

### TRACEABILITY

GLUCOSE STANDARD 100 and GLUCOSE STANDARD 300 are traceable to the SRM 965B reference material.

### WASTE MANAGEMENT

Please refer to local legal requirements.

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