CORMAY SERUM HN / CORMAY SERUM HP

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INTENDED USE

CORMAY SERUM HN / CORMAY SERUM HP is designed for routine quality control of inorganic, organic and enzymatic constituents.

The reagents must be used only for *in vitro* diagnostic, by suitably qualified laboratory personnel, only for the intended purpose, under appropriate laboratory conditions.

CHARACTERISTICS

CORMAY SERUM HN / CORMAY SERUM HP is lyophilised serum of human origin designed.

The declared values were calculated on the basis of results obtained in dozens of laboratories, on various analysers and by manual methods. For most parameters in CORMAY SERUM HN, the values are within the normal range for adults; in CORMAY SERUM HP, they are mostly pathological.

ASSIGNED VALUES

Assigned values for parameters, different for each batch, are given on the assay sheet included in the kit.

It is recommended that each laboratory establish its own means and acceptable ranges.

PACKAGE

CORMAY SERUM HN
SERUM HN 4 x 5 ml

CORMAY SERUM HP
SERUM HP 4 x 5 ml

RECONSTITUTION

- Open the vial carefully and add **exactly 5ml** of distilled water at 20-25°C.
- Replace the rubber stopper and leave to stand for **30 minutes** (in case of the ALP determinations stand for 1 hour) out of bright light.
- Swirl **gently** several times during reconstitution period to ensure that the contents are completely dissolved.
- Prior to use, mix the contents by inverting the vial several times.
- Do not shake! Avoid the foam formation!**

STABILITY AND STORAGE

- Lyophilised serum remains stable when stored at 2-8°C until expiry date given on the product label.
- The constituents in reconstituted control serum are stable for (except of parameters mentioned in notes):
 - Temp. 2-8°C 7 days
 - Temp. -20°C 30 days

WARNINGS AND NOTES

- Protect from direct sunlight and avoid contamination!
- This product should be treated the same as patient specimens and run in accordance with the instructions accompanying the reagent being used.
- When used for the quality control of ACP, ALAT, ASAT, ALP, BIL DIRECT determination, CORMAY SERUM HN/HP can be stored up to 3 days at 2-8°C after reconstitution.
- The stability of the control serum after reconstitution, intended for quality control of ACP activity assays, is 7 days at -20°C.
- Bilirubin as well as other organic compounds are light sensitive and it is recommended that the serum should be stored in the dark.
- Before alkaline phosphatase activity measurement allow the reconstituted or thawed serum to stand closed for 1 hour at 20-25°C to enable full reconstitution of the enzyme.
- In case of microbial contamination or excessive turbidity of the reconstituted serum do not use for assays quality control.
- This control serum is not intended for use as a calibrator.
- Only one freezing of the reconstituted serum is recommended! Preparation of portions and freezing should be done immediately after reconstitution. For freezing prepare portions of the serum no less than 500 µl in small (1.5 ml), tightly-closed vials. Thaw at room temperature for about 30 minutes (in case of the ALP determinations stand for 1 hour). Swirl gently several times during thawing.
- The controls are based on human serum that has been tested for HBsAg and antibodies to HIV and HCV and found to be non-reactive. However this material should be handled as thought capable of transmitting infectious disease.
- When the direct method for determination of total iron binding capacity (dTIBC) is used for internal quality control, we recommend the use of non-frozen control serum.
- EUH208 Contains gentamicin, sulfate (salt). May produce an allergic reaction.
- EUH210 Safety data sheet available on request.

WASTE MANAGEMENT

Please refer to legal requirements.

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