CORMAY URINALYSIS CONTROLS

CORMAY URINALYSIS CONTROLS 2 x 12 ml **Nr kat. 6-054**

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CORMAY URINALYSIS CONTROLS 2 x 12 ml **Cat. No 6-054**

INTENDED USE

CORMAY URINALYSIS CONTROLS is intended for use as an assayed quality control urine to monitor the precision of urinalysis test procedures.

Urine control should be used only for *in vitro* diagnostic, by suitably qualified laboratory personnel, only for the intended purpose, under appropriate laboratory conditions.

INTRODUCTION

CORMAY URINALYSIS CONTROL LEVEL 1 and LEVEL 2 is prepared from human urine with added human erythrocytes, simulated leukocytes, constituents of animal origin, chemicals, preservatives and stabilizers. The control is provided in liquid form.

PACKAGE

CORMAY URINALYSIS CONTROL LEVEL 1 1 x 12 ml
CORMAY URINALYSIS CONTROL LEVEL 2 1 x 12 ml

INSTRUCTION FOR USE

1. Remove the controls from refrigerator and allow to come to room temperature (18-25°C).
2. Swirl gently to assure homogeneity of the contents. Avoid foaming.
3. Apply directly on the strips using e.g. Pasteur's pipette.
4. Use controls as a patient sample in accordance with the requirements of the test method.
5. Immediately recap the controls and return to 2-8°C when not in use.

STABILITY AND STORAGE

1. Urinalysis controls remain stable until expiry date given on the product label when stored unopened at 2-8°C.
2. Store urinalysis controls tightly capped, avoid contamination.
3. Discard the controls if excessive turbidity or if there is any evidence of microbial contamination.
4. Product once opened and stored tightly capped at 2-8°C, will be stable for 30 days.

NOTES

- Use urine control taking into consideration analytical range of particular diagnostic kits for determination of parameters included in values sheet.
- The serum from each donor contributing urine for Urinalysis Controls were tested for the HIV, HBV antibody and HCV antibody and found to be non-reactive. However Urinalysis Controls should be handled as thought capable of transmitting infectious disease.
- This product is not intended for use as a standard.
- Contains gentamicin.
- Cormay Urinalysis Controls meeting the criteria for classification in accordance with Regulation (EC) No 1272/2008.

Ingredients:

5-chloro-2-methyl-2H-isothiazol-3-one

WARNING:

H317 May cause an allergic skin reaction.
P280 Wear protective gloves/protective clothing/eye protection/face protection.
P302 + P352 IF ON SKIN: Wash with plenty of soap and water.

P333 + P313 If skin irritation or rash occurs: Get medical advice.

P363 Wash contaminated clothing before reuse.

P261 Avoid breathing dust/fume/gas/mist/vapours/spray.

P272 Contaminated work clothing should not be allowed out of the workplace.

WASTE MANAGEMENT

Please refer to local legal requirements.

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