## LYSING REAGENT CN FREE

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### INTENDED USE

LYSING REAGENT CN FREE is intended to lyse of erythrocytes for hemoglobin release to allow further hemoglobin's assay on hematological analyzers MINDRAY BC-3000Plus. It prepares leukocytes for subpopulations differentiation process. It is only for *in vitro* diagnostics, for healthcare professional users.

### OPERATION PRINCIPLE

LYSING REAGENT CN FREE is located on board the analyzer.

Due to the lysing properties of the reagent, it is possible to disrupt the cell membrane of red blood cells and release cellular contents in preparation for further analysis. LYSING REAGENT CN FREE contains surfactants that lyses the red blood cells and leukoprotective agents that protects the state of the leukocytes to permit the differentiation in subpopulations.

### CONCENTRATION OF THE ACTIVE INGREDIENTS IN THE REAGENT

quaternary ammonium salts	2,87 %
sodium sulfate	
sodium chloride	
imidazole buffer stabilizers	

### REAGENT STABILITY

The reagent must be used before the expiry date indicated on the label.

### WARNINGS AND NOTES

- Protect from direct sunlight!
- Do not use reagent beyond expiry date printed on the label.
- Do not combine the remaining reagent with the reagent from a new package.
- Use personal protective equipment to prevent contact with samples, reagents and controls.
- Avoid discharge to sewage system or to environment.
- Please refer to the MSDS for detailed information concerning safe storage and use of the product.
- LYSING REAGENT CN FREE meeting the criteria for classification in accordance with Regulation (EC) No 1272/2008.

### Warning

H400 Very toxic to aquatic life.  
P273 Avoid release to the environment.  
P391 Collect spillage.

### DEVICES INTENDED TO BE USED IN COMBINATION WITH THE REAGENT

LYSING REAGENT CN FREE is intended to be used in combination with the diluent, cleaner and appropriate hematology analyzer.

Nazwa	Nr kat.
DILUENT	8-892
CLEANING REAGENT	8-894
ENZYMATIC CLEANER FORTE	8-895
MINDRAY BC-3000Plus	N/A

### ASSAY PROCEDURE

LYSING REAGENT CN FREE is ready for use. Prior the first use, unopened reagent must be stored for minimum 24 hours at room temperature. Follow the analyzer's operator manual.

### WASTE MANAGEMENT<sup>1</sup>

After use, the reagents should be handled as potentially infectious and disposed of in accordance with local legal requirements.

- Reagents residues: 18 01 06\*
- Empty packages: 15 01 10\*
- Wastewater from the analyzer: 18 01 03\*

### INCIDENTS<sup>2</sup>

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer (website address: [incidents@cormay.pl](mailto:incidents@cormay.pl)) and the competent authority of the Member State in which the user and/or the patient is established.

**Serious incident** means any incident that directly or indirectly led, might have led or might lead to any of the following:

- the death of a patient, user or other person,
- the temporary or permanent serious deterioration of a patient's, user's or other person's state of health,
- a serious public health threat.

### LITERATURE

- European Commission notice on technical guidance on the classification of waste (2018/C 124/01) of 9 April 2018.
- Regulation (EU) 2017/746 of the European Parliament and of the Council of 5 April 2017 on *in vitro* diagnostic medical devices.

### LIST OF CHANGES

Previous version: 51_03_13_003_01	Current version: 51_03_13_016_01
--------------------------------------	-------------------------------------

Creation the new format of the document.

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