Cystatin C Control Set

Liquid human based controls for use as controls in immunoturbidimetric assays for Cystatin C

REF	Cont.		
906880	6x 2 mL	3x 2 mL Control Low 3x 2 mL Control High	Cystatin C Control Set

COMPOSITION

Dialab Cystatin C Control Set is human serum based.
 Natriumazide 0.095 %

PREPARATION

The Cystatin C Control Set is liquid, ready to use.

STABILITY AND STORAGE

Storage: at 2 – 8 °C
 Stability: up to the expiration date
 Stability after opening: 4 weeks at 2 – 8 °C
DO NOT FREEZE!

DIRECTIONS FOR USE

- Use these controls in the same manner as specified with the reagent and/or instrument being used.
- Allow the control to equilibrate to room temperature (22 – 28 °C) prior to use.
- The control should not be allowed to stand for repeated long periods of time (up to 6 hours) at room temperature. Keep vials tightly capped at all times when not in use to avoid microbial contamination. Keep vials refrigerated when not in use.

WARNINGS AND PRECAUTIONS

- For in Vitro Diagnostic use.
- Each individual donation intended for use in manufacture of protein calibrator was tested for hepatitis B surface antigen (HBsAg), anti-hepatitis C (anti-HCV) and anti-HIV 1 and HIV 2 by FDA required tests. Since no test method can assure that products derived from human blood do not contain HIV-1/2 and Hepatitis B and Hepatitis C virus, this material and all patient samples should be handled as though capable of transmitting infectious diseases.
- Reagents containing sodium azide must be handled with due caution: Do not ingest or allow to contact skin or mucous membranes! Sodium azide can form explosive azides when contacting heavy metals such as copper or lead.

WASTE MANAGEMENT

Please refer to local legal requirements.

LOT SPECIFIC ASSAY DATA

Values and expiry date are lot specific.

LOT: 456091		EXP: 2018-05
Parameter	Target Value	Range
Cystatin C Control Low	0.80 mg/L	0.70 – 0.90 mg/L
Cystatin C Control High	4.00 mg/L	3.50 – 4.50 mg/L

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