## D-Dimer Calibrator 5 Level Series

Lyophilized human based calibrators for use as calibrators in immunoturbidimetric assays for D-Dimer

**REF**

913702

**Cont.**

5x 1 mL

D-Dimer Calibrator 5 Level Series (Iyo.)

### COMPOSITION

The Dialab D-Dimer Calibrator Series is based on human serum and additives. For in vitro diagnostic use only.

Sodium azide 0.095 %

### PREPARATION

The D-Dimer Calibrator Series is supplied in lyophilized form (5x 1 mL).

### STABILITY AND STORAGE

Storage: at 2 – 8 °C  
 Stability: up to the expiration date  
 Stability after reconstitution: 14 days at 2 – 8 °C  
 DO NOT FREEZE! Store tightly capped when not in use.

### DIRECTIONS FOR USE

1. Open the vial carefully, avoiding any loss of lyophilized material.
2. Add 1.0 mL of distilled water to the vial. Take care to avoid the formation of bubbles.
3. Cap and leave at room temperature for 30 minutes, inverting gently to dissolve contents. Mix carefully before each use.
4. Use in the same way as samples.
5. A saline solution is required to generate a 6-point calibration curve.

### WARNINGS AND PRECAUTIONS

1. For in Vitro Diagnostic use.
2. Each individual donation intended for use in manufacture of protein calibrator was tested for hepatitis B surface antigen (HBsAg), anti-hepatitis C (anti-HCV) and anti-HIV 1 and HIV 2 by FDA required tests. Since no test method can assure that products derived from human blood do not contain HIV-1/2 and Hepatitis B and Hepatitis C virus, this material and all patient samples should be handled as though capable of transmitting infectious diseases.
3. Reagents containing sodium azide must be handled with due caution: Do not ingest or allow to contact eyes, skin or mucous membranes! Sodium azide can form explosive azides when contacting heavy metals such as copper or lead.

### MATERIALS REQUIRED BUT NOT PROVIDED

- DIALAB D-Dimer (turb.) Assay (REF 913700)
- Distilled water
- 0.9 % saline solution

### REFERENCES

1. BJH Guideline. British Journal of Haematology. 124, 15-25.
2. Alan H.B. Wu. Tietz Clinical Guide to Laboratory Tests. Fourth Ed. Saunders Elsevier, 11830 Westline Industrial Drive, St Louis, Missouri, 63146. 2006; 328-329

### LOT SPECIFIC ASSAY DATA

Values and expiry date are lot specific. The D-Dimer concentrations are expressed in µg/mL FEU.

LOT: DDS001130-01-01		EXP: 2014/10
Parameter	Level	Assigned Value
D-Dimer	Level 1	0.60 µg/mL FEU
D-Dimer	Level 2	1.30 µg/mL FEU
D-Dimer	Level 3	2.50 µg/mL FEU
D-Dimer	Level 4	6.65 µg/mL FEU
D-Dimer	Level 5	8.00 µg/mL FEU

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