Lyophilized Control Serum

ADA CONTROL SET

2 Levels

Control serum for the quality control of the quantitative in vitro determination of adenosine deaminase (ADA) in human serum or plasma on photometric systems.

REF

Cont.

913880 **2 x 1 ml** Single vial

Additionally offered:

913816 6 x 25 ml ADA reagent kit
 913813 3 x 25 ml ADA reagent kit

CONTROL COMPOSITION

Bovine serum and additives.
 Sodium azide (NaN₃) < 0.1 %

CONTROL PREPARATION

1. Open one vial carefully to avoid any loss of material and reconstitute with exactly 1.0 mL of distilled water
2. Close the vial and let stand for thirty minutes at room temperature, dissolving contents completely by gently swirling or rotating.

CONTROL STABILITY AND STORAGE

Storage temperature: -20 °C
 Stability: until the expiration date

Stability of the reconstituted calibrator:

Storage temperature: 2 – 8 °C
 Stability: 1 week

Store the calibrator tightly capped when not in use.

WARNINGS AND PRECAUTIONS

1. For laboratory use only.
2. Product contains highly purified bovine source material from non-BSE countries. The manufacturing facility does not receive, store or process ruminant materials from restricted countries.
3. Possible infectious agents in the materials have been inactivated. Because no method can offer complete assurance as to the absence of infectious agents, this material should be handled as though capable of transmitting infectious disease and disposed as biohazard waste or medical waste according to applicable local and national laws.
4. Additional safety information concerning storage and handling of this product is provided within the Material Safety Data Sheet (MSDS) for this product. The MSDS is available on request.
5. Avoid contact with skin and eyes
6. Contains sodium azide (< 0.1%) which may react with lead or copper plumbing to form explosive compounds. Flush drains with copious amounts of water when disposing of this reagent.

WASTE MANAGEMENT

Please refer to local legal requirements.

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