ISE REAGENTS PACK

CALBRATORS FOR DETERMINATION OF ELECTROLYTES CONCENTRATION

ISE REAGENTS PACK

Cat. No E-701

WASTE MANAGEMENT

Please refer to local legal requirements.

INTENDED USE

ISE REAGENTS PACK is intended for calibration and determinations of sodium, potassium, chloride, ionized calcium and lithium in ISE Electrolyte Analysers.

Date of issue: 04. 2010.

REAGENTS

Each fluid pack ISE REAGENTS PACK contains:

STANDARD A	1 x 350 ml
STANDARD B	1 x 85 ml
STANDARD C	1 x 85 ml
REFERENCE SOLUTION	1 x 100 ml

Active Ingredients

		Standard A	Standard B	Standard C	Reference Solution
Sodium	mmol/L	150.0	100.0	150.0	-
Potassium	mmol/L	5.0	1.8	5.0	-
Chloride	mmol/L	115.0	72.0	115.0	-
Calcium	mmol/L	0.9	1.5	0.9	-
Lithium	mmol/L	0.3	0.3	1.4	-
Potassium Chloride	mol/L	-	-	-	1.2

STORAGE

- Reagents are stable when stored at 18-25°C until the expiration date printed on the label.
- Stability on board: 14 weeks.

INSTALLATION

- Press **YES** after the prompt **OPERATION FUNCTIONS** is displayed.
- Press **YES** after the prompt **Check/Change Fluid Pack?** is displayed.
- Press **YES** after the prompt **Change ISE Pack?** is displayed.
- Grasp the pack firmly and pull outward. If removal is difficult, press on the end of the ISE REAGENTS PACK guide pin located to the left of the measuring chamber inside the front cover.
- Install new ISE REAGENTS PACK:
 - Before use, remove the red plastic caps from the connector nipples.
 - Save caps to close the connector nipples of used ISE REAGENTS PACK.
 - Write the date of installation on the label side of the ISE REAGENTS PACK with a permanent marker.
 - Press the new pack firmly into the cavity on the left side of the instrument while holding the right side of instrument for stability.
 - The analyser will display the amount of remaining fluid. Then, the prompt **New Fluid Pack Installed?** will appear. Press **YES** to indicate that a new reagent pack is installed. The analyser will prompt **Are you sure?** Press **YES**.
 - Analyzer will automatically reset the reagents pack counter to 100% and commence system calibration

NOTES

- Product for in vitro diagnostic use only.
- Do not freeze.
- The ISE REAGENTS PACK **after use** contains human source material. Therefore this material should be handled as though capable of transmitting infectious disease.
- DO NOT turn the instrument power OFF with the ISE REAGENTS PACK installed, as this can cause contamination of the unused standard solutions remaining in the pack.
- ISE REAGENTS PACK should be used by suitably qualified laboratory personnel only in accordance with intended purpose.

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PZ CORMAY S.A.