



DRG® CatCombi ELISA (EIA-3175)



REVISED 7 SEPT. 2011 RM (VERS. 5.1)

RUO IN THE USA

This kit is intended for Research Use Only.

Not for use in diagnostic procedures.

Please use only the valid version of the package insert provided with the kit.

INTENDED USE

Manual and automated enzyme immunoassay for determination of adrenalin (epinephrine) and noradrenalin (norepinephrine) in human plasma and urine.

TEST PRINCIPLE

Solid phase enzyme-linked immunosorbent assay (ELISA) based on the sandwich principle. The wells are coated with a goat anti rabbit antibody. The added liquid antibody, directed towards an epitope of an antigen molecule binds to the plate within the incubation time. The antigen of the sample is incubated in the coated well with enzyme conjugated second antibody (E-Ab), directed towards a different region of the antigen molecule. After the substrate reaction the intensity of the developed color is proportional to the amount of the antigen. Results of samples can be determined directly using the standard curve.

WARNINGS AND PRECAUTIONS

1. For research use only.
2. Before starting the assay, read the instructions completely and carefully. Use the valid version of the package insert provided with the kit. Be sure that everything is understood.
3. In case of severe damage of the kit package please contact DRG or your supplier in written form, latest one week after receiving the kit. Do not use damaged components in test runs, but keep safe for complaint related issues.
4. Obey lot number and expiry date. Do not mix reagents of different lots. Do not use expired reagents.
5. Follow good laboratory practice and safety guidelines. Wear lab coats, disposable latex gloves and protective glasses where necessary.
6. Reagents of this kit containing hazardous material may cause eye and skin irritations. See MATERIALS SUPPLIED and labels for details. Material Safety Data Sheets for this product are available upon request.
7. Chemicals and prepared or used reagents have to be treated as hazardous waste according to national biohazard and safety guidelines or regulations.
8. Avoid contact with Stop solution. It may cause skin irritations and burns.
9. All reagents of this kit containing human serum or plasma have been tested and were found negative for anti-HIV I/II, HBsAg and anti-HCV. However, a presence of these or other infectious agents cannot be excluded absolutely and therefore reagents should be treated as potential biohazards in use and for disposal.

STORAGE AND STABILITY

The kit is shipped at ambient temperature and should be stored at 2-8°C. Keep away from heat or direct sun light. The storage and stability of specimen and prepared reagents is stated in the corresponding chapters.

The microtiter strips are stable up to the expiry date of the kit in the broken, but tightly closed bag when stored at 2-8°C.

SPECIMEN COLLECTION AND STORAGE

The in-vivo catecholamine and metanephrines release is influenced by several foods and drugs. Vitamin B, coffee and bananas, alpha-methyldopa, MAO and COMT inhibitors as well as medications related to hypertension should be discontinued for at least 72 h prior to specimen collection.

Plasma (EDTA)

The blood sample should be stored at 2-8°C until centrifuged to separate the plasma within 2 h after blood collection.

The usual precautions for venipuncture should be observed. It is important to preserve the chemical integrity of a blood specimen from the moment it is collected until it is assayed. Do not use grossly hemolytic, icteric or grossly lipemic specimens. Samples appearing turbid should be centrifuged before testing to remove any particulate material.

Storage:	2-8°C	≤ -20°C (Aliquots)	Keep away from heat or direct sun light. Avoid repeated freeze-thaw cycles. Ship samples frozen.
Stability:	6 h	1 mon	

Urine

It is possible to use spontaneous as well as 24 h urine. The total volume of urine excreted during a 24 h period should be collected and mixed in a single bottle containing 10-15 mL of 6 N HCl as preservative. Determine total volume for calculation of results. **Mix and centrifuge samples before use in the assay.**


Storage:	≤ -20°C (Aliquots)	Keep away from heat or direct sun light. Avoid repeated freeze-thaw cycles.
Stability:	6 mon	

MATERIALS SUPPLIED

The reagents provided with this kit are sufficient for 96 extractions in single determinations in the sample preparation (extraction): 88 specimen samples, 6 standards and 2 controls. Each extract is sufficient for a single determination for adrenalin and noradrenalin immunoassay.



The microtiter plate can be used for Adrenalin and Noradrenalin.

Quantity	Symbol	Component
2 x 12x8		Microtiter Plate Break apart strips. Coated with anti-rabbit IgG (goat, polyclonal).

1 x 6 x 2.5 mL	CAL A-F	Standard A-F Adrenalin: 0; 1.5; 5.0; 15; 50; 150 ng/mL (0; 8; 27; 82; 273; 819 nmol/L) Noradrenalin: 0; 5.0; 15; 50; 150; 500 ng/mL (0; 30; 89; 296; 887; 2955 nmol/L) Dopamine: 0; 60; 180; 585; 2300; 11470 ng/mL (0; 392; 1175; 3819; 15014; 74876 nmol/L) Ready to use. Contains: [-] Adrenalin, [-] Noradrenalin [-] Dopamine (biologically active), 0.1 M HCl.
1 x 2 x 2.5 mL	CONTROL 1+2	Control 1+2 Ready to use. Contains: [-] Adrenalin, [-] Noradrenalin, [-] Dopamine (biologically active), 0.1 M HCl. Concentrations / acceptable ranges see QC Certificate.
2 x 250 µL	ENZCONJ CONC	Enzyme Conjugate Concentrate (100x) Contains: antibodies, conjugated to alkaline phosphatase, Tris buffer, HCl, 0.01 % NaN ₃ .
4 x	EXTRPLATE	Extraction Plate (Macrotiter Plate) 24 wells each. Coated with boronate affinity gel.
2 x 60 mL	EXTRBUF	Extraction Buffer Pink colored. Ready to use. Contains: 0.016 % NaN ₃ .
4 x 1.25 mL	COMT LYO	COMT lyophilized Contains: Catechol-O-methyltransferase (porcine liver), NaN ₃ .
4 x 1.25 mL	COENZ	Coenzyme Solution Ready to use. Contains: S-Adenosyl-L-Methionine, stabilizers.
2 x 3 mL	ENZBUF	Enzyme Buffer Ready to use. Contains: Tris buffer, HCl, stabilizers.
1 x 100 mL	RELEASEBUF	Release Buffer Yellow Colored. Ready to use. Contains: 0.1 M HCl, indicator.
2 x 3 mL	ACYLREAG	Acylation Reagent Ready to use. Contains: dimethylformamide, Ethanol. Caution! Toxic. Highly flammable.
3 x 50 mL	WASHBUF CONC	Wash Buffer Concentrate (10x) Contains: Tris buffer, HCl, Tween, 0.2 % NaN ₃ .
1 x 2 mL	COMT ADD	COMT Additive Contains: human plasma, stabilizers, 0.01 % Thimerosal.
1 x 7.0 mL	ANTISERUM AD	Adrenalin Antiserum Green colored Ready to use. Contains: antibodies against Adrenalin (rabbit), Buffer, stabilizers.

1 x 7.0 mL	ANTISERUM NAD	Noradrenalin Antiserum Blue colored. Ready to use. Contains: antibodies against Noradrenalin (rabbit), Buffer, stabilizers.
2 x 9 x	PNPP SUBS	PNPP Substrate Tablets In one foil packet. Contains: p-nitrophenyl phosphate (PNPP).
2 x 27 mL	PNPP BUF	PNPP Substrate Buffer Ready to use. Contains: diethanolamine, water, 0.05 % NaN ₃ .
2 x 15 mL	PNPP STOP	PNPP Stop Solution Ready to use. Contains: 1 M NaOH, 0.25 M EDTA.
6 x	FOIL	Adhesive Foil

MATERIALS REQUIRED BUT NOT SUPPLIED

1. Micropipettes (Multipette Eppendorf or similar devices, < 3% CV). Volume: 10; 10-100; 100-1000 µL
2. Orbital shaker (200-900 rpm) (e.g. EAS 2/4, SLT)
3. Vortex mixer
4. 8-Channel Micropipettor with reagent reservoirs
5. Wash bottle, automated or semi-automated microtiter plate washing system
6. Microtiter plate reader capable of reading absorbance at 405 nm (reference wavelength 600-650 nm)
7. Bidistilled or deionised water
8. Paper towels, pipette tips and timer
9. Disposable tubes for sample dilution
10. 0.1 M HCl, for sample dilution (Urine)

PROCEDURE NOTES

1. Any improper handling of samples or modification of the test procedure may influence the results. The indicated pipetting volumes, incubation times, temperatures and pretreatment steps have to be performed strictly according to the instructions. Use calibrated pipettes and devices only.
2. Once the test has been started, all steps should be completed without interruption. Make sure that required reagents, materials and devices are prepared ready at the appropriate time. Allow all reagents and specimens to reach room temperature (18-25°C) and gently swirl each vial of liquid reagent and sample before use. Mix reagents without foaming.
3. Avoid contamination of reagents, pipettes and wells/tubes. Use new disposable plastic pipette tips for each component and specimen. Do not interchange caps. Always cap not used vials. Do not reuse wells/tubes or reagents.
4. Some components contain $\leq 250 \mu\text{L}$ solution. Take care that the solution is completely on the bottom of the vial before opening.
5. It is advised to determine samples in duplicate to be able to identify potential pipetting errors.
6. Use a pipetting scheme to verify an appropriate plate layout. A pipetting scheme covering both sample pretreatment and assay is available upon request.
7. Incubation time affects results. All wells should be handled in the same order and time sequences. It is recommended to use an 8-channel Micropipettor for pipetting of solutions in all wells.
8. Microplate washing is important. Improperly washed wells will give erroneous results. It is recommended to use a multichannel pipette or an automatic microplate washing system. Do not allow the wells to dry between incubations. Do not scratch coated wells during rinsing and aspiration. Rinse and fill all reagents with care. While rinsing, check that all wells are filled evenly with Wash Buffer, and that there are no residues in the wells.
9. Humidity affects the coated wells/tubes. Do not open the pouch until it reaches room temperature. Unused wells/tubes should be returned immediately to the resealed pouch including the desiccant.

MANUAL PROCEDURE

PRE-TEST SETUP INSTRUCTIONS

	The contents of the kit for 192 determinations can be divided into 2 separate runs.
	Visible amounts of gel can be separated from surface of extraction plate during extraction. This has no influence on test results.
	Air contamination by peroxygen containing disinfectants for cleaning of surfaces or equipment used as powder or as solutions, e.g. VIRKON[®] must be avoided in any case. They will strongly disturb assay performance. VIRKON[®] is a trademark of DuPont.

Dilution of Samples

Samples suspected to contain concentrations above the highest standard have to be diluted as follows:

Sample	to be diluted	with	Remarks
Plasma	> highest standard	bidist. water	prior to extraction step
Urine	> highest standard	0.1 N HCl	prior to extraction step

Extraction of Samples, Standards and Controls (Extraction Plate) (manual version)

1.	Pipette 20 µL of each Standard, Control and urine sample and 500 µL of each plasma sample into the respective wells of the extraction plate. Add 500 µL of bidist. water to all wells except for the plasma samples to correct differences of volumes.
2.	Pipette 1000 µL of Extraction Buffer into each well.
3.	Cover plate with adhesive foil. Extract 30 min at RT (18-25°C) on an orbital shaker (600–900 rpm). During extraction the surface of the liquid should wet the adhesive foil, but the liquid level should not exceed 2/3 of the well. Splashing does not affect results.
4.	Remove adhesive foil. Immediately empty plate and eliminate residual fluid on a paper towel.
5.	Pipette 2 mL of bidist water into each well.
6.	Cover plate with new adhesive foil. Shake 5 min at RT (18-25°C) on an orbital shaker (600–900 rpm). Splashing does not affect results.
7.	Remove adhesive foil. Immediately empty plate and eliminate residual fluid on a paper towel. Remove fluid completely.
8.	Pipette 150 µL of Extraction Buffer into each well. To each well add 50 µL of Acylation Reagent . Mix immediately after pipetting.
9.	Extract 20 min at RT (18-25°C) (without adhesive foil) on an orbital shaker (400–600 rpm).
10.	Immediately empty plate and eliminate residual fluid on a paper towel. Remove fluid completely.
11.	Pipette 2 mL of bidist. water into each well.
12.	Cover plate with new adhesive foil. Shake 5 min at RT (18-25°C) on an orbital shaker (600–900 rpm). Splashing does not affect results.



DRG® CatCombi ELISA (EIA-3175)




REVISED 7 SEPT. 2011 RM (VERS. 5.1)

RUO IN THE USA

13.	Remove adhesive foil. Immediately empty plate and eliminate residual fluid on a paper towel. Remove fluid completely.
14.	Pipette 300 µL of Release Buffer into each well.
15.	Shake 30 min at RT (18-25°C) (without adhesive foil) on an orbital shaker (400–600 rpm).

Prepared samples should be assayed the same day. If this is not possible, you can store the extraction plate covered with adhesive foil at 2-8°C over night.

Preparation of lyophilized or concentrated components

	The volumes stated below are for one run with 2 x 6 strips (2 x 48 determinations)
--	--

Dilute / dissolve	Component	with	Diluent	Relation	Remarks	Storage	Stability
25 mL	WASHBUF CONC	225 mL	bidist. water	1:10	Mix vigorously.	2-8°C	4 w
110 µL	ENZCONJ CONC	11 mL	WASHBUF (diluted)	1:101	Prepare freshly and use only once. Mix without foaming.	18-25°C	5 h
9	PNPP SUBS	24 mL	PNPP BUF		Prepare freshly and use only once.	18-25°C	5 h




REVISED 7 SEPT. 2011 RM (VERS. 5.1)

RUO IN THE USA

TEST PROCEDURE (manual version)

Preparation of COMT Enzyme Solution


 **The COMT Enzyme Solution should be freshly prepared directly before use.**

Dissolve each kit component of lyophilized **COMT** in **1.25 mL bidist. water** and mix the dissolved COMT.* Then **pipette 1.25 mL of Coenzyme Solution** followed by **1.25 mL of Enzyme Buffer** and **0.40 mL COMT Additive** to the mixed COMT vials to give a final volume of 4.15 mL of COMT Enzyme Solution per vial.

Pool two (2) vials for 48 determinations of adrenalin and 48 determinations of noradrenalin. Solution may be turbid. Mix without foaming. The COMT solution is stable at room temperature for 1 hour.

* If only an aliquot of the COMT solution is needed, the rest of the COMT solution should be frozen immediately in aliquots at -20° C. The COMT solution is stable under these conditions for 1-2 months.

Enzymatic Derivatization of Samples, Standards and Controls (Microtiter Plate)

 It is recommended to start with adrenalin.

If pipetting with *positive displacement*, give the residual fluid from the tip of the pipette back to the corresponding wells of the extraction plate, otherwise the extracts may not be sufficient for noradrenalin determination.

It is useful to hold the extraction plate in a sloping position.

Before use of the Microtiter plates, define and label the wells for Adrenalin and Noradrenalin.

For research use of tissue homogenates and cell culture supernatants a general recommendation can be given:

Working with cell culture supernatants depends on the matrix as well as concentrations expected:

According to the urine protocol (extraction of at least 20 µL supernatant) a sensitivity for adrenalin of 0.3 ng/mL and for noradrenaline of 0.6 ng/mL for diluted sample can be expected. In case of a matrix with addition of serum (FCS), plasma protocol (extraction of 500 µL supernatant) can be used with the sensitivities corresponding to the plasma protocol (see 16. PERFORMANCE).

For tissue homogenates no perchloric acid should be used for homogenization. For further details ask DRG.

Adrenalin for urine and plasma

1.	Pipette 75 µL of freshly prepared COMT Enzyme Solution into each well of the Microtiter Plate . Shake plate briefly.
2.	Pipette 100 µL of each <u>extracted Standard, Control and sample</u> into the respective wells. During this step hold the pipette tips directly into the COMT solution. Color changes to pink. Shake briefly.
3.	Pipette 50 µL of Adrenalin Antiserum (green colored) into each well.
4.	Cover plate with adhesive foil. Incubate 120 min at RT (18-25°C) on an orbital shaker (400–600 rpm).

Noradrenalin for urine and plasma

1.	Pipette 25 µL of freshly prepared COMT Enzyme Solution into each well of the Microtiter Plate . Shake plate briefly.
2.	Pipette 25 µL of each <u>extracted</u> Standard, Control and sample into the respective wells. During this step hold the pipette tips directly into the COMT solution. Color changes to pink. Shake plate briefly.
3.	Pipette 50 µL of Noradrenalin Antiserum (blue colored) into each well.
4.	Cover plate with adhesive foil. Incubate 120 min at RT (18-25°C) on an orbital shaker (400–600 rpm).

ELISA

The following procedure must be performed for Adrenalin and Noradrenalin.

1.	Remove adhesive foil. Discard incubation solution. Wash plate 4 x with 250 - 300 µL of diluted Wash Buffer . Remove excess solution by tapping the inverted plate on a paper towel.
2.	Pipette 100 µL of freshly prepared Enzyme Conjugate into each well.
3.	Cover plate with new adhesive foil. Incubate 60 min at RT (18-25°C) on an orbital shaker (400–600 rpm).
4.	Remove adhesive foil. Discard incubation solution. Wash plate 4 x with 250 - 300 µL of diluted Wash Buffer . Remove excess solution by tapping the inverted plate on a paper towel.
5.	For adding of Substrate and Stop Solution use, if available, an 8-channel Micropipettor. Pipetting should be carried out in the same time intervals for Substrate and Stop Solution. Use positive displacement and avoid formation of air bubbles.
6.	Pipette 200 µL of PNPP Substrate Solution into each well.
7.	Incubate 40 min at RT (18-25°C) (without adhesive foil) on an orbital shaker (400–600 rpm).
8.	Stop the substrate reaction by adding 50 µL of PNPP Stop Solution into each well. Briefly mix contents by gently shaking the plate.
9.	Measure optical density with a photometer at 405 nm (Reference-wavelength: 620-650 nm) within 60 min after pipetting of the Stop Solution. No air bubbles should be visible.

AUTOMATED PROCEDURE

PRE-TEST SETUP INSTRUCTIONS (automated version)

	The contents of the kit for 192 determinations can be divided into 2 separate runs.
	Visible amounts of gel can be separated from surface of extraction plate during extraction. This has no influence on test results.
	Air contamination by peroxygen containing disinfectants for cleaning of surfaces or equipment used as powder or as solutions, e.g. VIRKON[®] must be avoided in any case. They will strongly disturb assay performance. VIRKON[®] is a trademark of DuPont.

Dilution of Samples

Samples suspected to contain concentrations above the highest standard have to be diluted as follows:

Sample	to be diluted	with	Remarks
Plasma	> highest standard	bidist. water	prior to extraction step
Urine	> highest standard	0.1 N HCl	prior to extraction step

Extraction of Samples, Standards and Controls (Extraction Plate) (automated version)

1.	Pipette 30 µL of each Standard, Control and urine sample and 750 µL of each plasma sample into the respective wells of the extraction plate. Add 750 µL of bidist. water to all wells except for the plasma samples to correct differences of volumes.
2.	Pipette 1000 µL of Extraction Buffer into each well.
3.	Cover plate with adhesive foil. Extract 30 min at RT (18-25°C) on an orbital shaker (600–900 rpm). During extraction the surface of the liquid should wet the adhesive foil, but the liquid level should not exceed 2/3 of the well. Splashing does not affect results.
4.	Remove adhesive foil. Immediately empty plate and eliminate residual fluid on a paper towel.
5.	Pipette 2 mL of bidist water into each well.
6.	Cover plate with new adhesive foil. Shake 5 min at RT (18-25°C) on an orbital shaker (600–900 rpm). Splashing does not affect results.
7.	Remove adhesive foil. Immediately empty plate and eliminate residual fluid on a paper towel. Remove fluid completely.
8.	Pipette 150 µL of Extraction Buffer into each well. To each well add 50 µL of Acylation Reagent . Mix immediately after pipetting.
9.	Extract 20 min at RT (18-25°C) (without adhesive foil) on an orbital shaker (400–600 rpm).
10.	Immediately empty plate and eliminate residual fluid on a paper towel. Remove fluid completely.
11.	Pipette 2 mL of bidist. water into each well.
12.	Cover plate with new adhesive foil. Shake 5 min at RT (18-25°C) on an orbital shaker (600–900 rpm). Splashing does not affect results.
13.	Remove adhesive foil. Immediately empty plate and eliminate residual fluid on a paper towel. Remove fluid




REVISED 7 SEPT. 2011 RM (VERS. 5.1)

RUO IN THE USA

	completely.
14.	Pipette 450 µL of Release Buffer into each well.
15.	Shake 30 min at RT (18-25°C) (without adhesive foil) on an orbital shaker (400–600 rpm).

Prepared samples should be assayed the same day. If this is not possible, you can store the extraction plate covered with adhesive foil at 2-8°C over night.

Preparation of lyophilized or concentrated components

	The volumes stated below are for one run with 2 x 6 strips (2 x 48 determinations)
--	--

Dilute / dissolve	Component	with	Diluent	Relation	Remarks	Storage	Stability
50 mL	WASHBUF CONC	450 mL	bidist. water	1:10	Mix vigorously.	2-8°C	4 w
150 µL	ENZCONJ CONC	15 mL	WASHBUF (diluted)	1:101	Prepare freshly and use only once. Mix without foaming.	18- 25°C	5 h
9	PNPP SUBS	24 mL	PNPP BUF		Prepare freshly and use only once.	18- 25°C	5 h

TEST PROCEDURE (automated version)**Preparation of COMT Enzyme Solution**

The COMT Enzyme Solution should be freshly prepared directly before use.

Dissolve each kit component of lyophilized **COMT** in **1.25 mL bidist. water** and mix the dissolved COMT.*

Then **pipette 1.25 mL of Coenzyme Solution** followed by **1.25 mL of Enzyme Buffer** and **0.40 mL COMT Additive** to the mixed COMT vials to give a final volume of 4.15 mL of COMT Enzyme Solution per vial.

Pool two (2) vials for 48 determinations of adrenalin and 48 determinations of noradrenalin. Solution may be turbid. Mix without foaming. The COMT solution is stable at room temperature for 1 hour.

* If only an aliquot of the COMT solution is needed, the rest of the COMT solution should be frozen immediately in aliquots at -20° C. The COMT solution is stable under these conditions for 1-2 months.

For research use of tissue homogenates and cell culture supernatants a general recommendation can be given:

Working with cell culture supernatants depends on the matrix as well as concentrations expected:

According to the urine protocol (extraction of at least 20 µL supernatant) a sensitivity for adrenalin of 0.3 ng/mL and for noradrenaline of 0.6 ng/mL for diluted sample can be expected. In case of a matrix with addition of serum (FCS), plasma protocol (extraction of 500 µL supernatant) can be used with the sensitivities corresponding to the plasma protocol (see 16. PERFORMANCE).

For tissue homogenates no perchloric acid should be used for homogenization. For further details ask DRG.

Enzymatic Derivatization of Samples, Standards and Controls (Microtiter Plate)**Adrenalin for urine and plasma**

1.	Pipette 75 µL of freshly prepared COMT Enzyme Solution into each well of the Microtiter Plate . Shake plate 1 min.
2.	Pipette 100 µL of each <u>extracted</u> Standard, Control and sample into the respective wells. Shake plate 1 min.
3.	Pipette 50 µL of Adrenalin Antiserum (green colored) into each well.
4.	Cover plate. Incubate 120 min at RT (18-25°C) on an orbital shaker (400–600 rpm).

Noradrenalin for urine and plasma

1.	Pipette 25 µL of freshly prepared COMT Enzyme Solution into each well of the Microtiter Plate . Shake plate 1 min.
2.	Pipette 25 µL of each <u>extracted</u> Standard, Control and sample into the respective wells. Shake plate 1 min.
3.	Pipette 50 µL of Noradrenalin Antiserum (blue colored) into each well.
4.	Cover plate. Incubate 120 min at RT (18-25°C) on an orbital shaker (400–600 rpm).

ELISA (automated version)

The following procedure must be performed for Adrenalin and Noradrenalin.

1.	Discard incubation solution. Wash plate 6 x with 250 - 300 µL of diluted Wash Buffer .
2.	Pipette 100 µL of Enzyme Conjugate into each well.
3.	Cover plate. Incubate 60 min at RT (18-25°C) on an orbital shaker (400–600 rpm).
4.	Discard incubation solution. Wash plate 6 x with 250 - 300 µL of diluted Wash Buffer .
5.	Pipetting should be carried out in the same time intervals for Substrate and Stop Solution.
6.	Pipette 200 µL of PNPP Substrate Solution into each well.
7.	Incubate 40 min at RT (18-25°C) on an orbital shaker (400–600 rpm). <i>If temperature in automat exceeds 25°C, shorten incubation time to 30 min to avoid signal overflow.</i>
8.	Stop the substrate reaction by adding 50 µL of PNPP Stop Solution into each well. Briefly mix contents by gently shaking the plate.
9.	Measure optical density with a photometer at 405 nm (Reference-wavelength: 620-650 nm) within 60 min after pipetting of the Stop Solution.

QUALITY CONTROL

The test results are only valid if the test has been performed following the instructions. Moreover the user must strictly adhere to the rules of GLP (Good Laboratory Practice) or other applicable standards/laws. All kit controls must be found within the acceptable ranges as stated on the labels and the QC certificate. If the criteria are not met, the run is not valid and should be repeated. Each laboratory should use known samples as further controls. It is recommended to participate at appropriate quality assessment trials.

In case of any deviation the following technical issues should be proven: Expiration dates of (prepared) reagents, storage conditions, pipettes, devices, incubation conditions and washing methods.

CALCULATION OF RESULTS

The obtained OD of the standards (y-axis, linear) are plotted against their concentration (x-axis, logarithmic) either on semi-logarithmic graph paper or using an automated method. A good fit is provided with cubic spline, 4 Parameter Logisites or Logit-Log.

For the calculation of the standard curve, apply each signal of the standards.

The concentration of the kit Controls and of the urine samples can be read directly from the corresponding standard curve. Due to the pipetting volume of 500 µL (automated version: 750 µL) for plasma in comparison to 20 µL (automated version: 30 µL) for the standards, the results for plasma samples have to be divided by 25. For units in pg/mL please multiply by 1000.

In case of diluted samples the values have to be multiplied with the corresponding dilution factor.

Samples showing concentrations above the highest standard have to be diluted as described in PRE-TEST SETUP INSTRUCTIONS and reassayed.

DRG® CatCombi ELISA (EIA-3175)



REVISED 7 SEPT. 2011 RM (VERS. 5.1)

RUO IN THE USA

Calculate the 24 h excretion for each urine sample: $\mu\text{g}/24 \text{ h} = \mu\text{g}/\text{L} \times \text{L}/24 \text{ h}$

Conversion:

1000 pg/mL = 1 ng/mL


Adrenalin ($\mu\text{g}/\text{L}$) x 5.458 = nmol/L

1000 pg/mL = 1 ng/mL


Noradrenalin ($\mu\text{g}/\text{L}$) x 5.911 = nmol/L

Typical Calibration Curve

(Example. Do not use for calculation!)

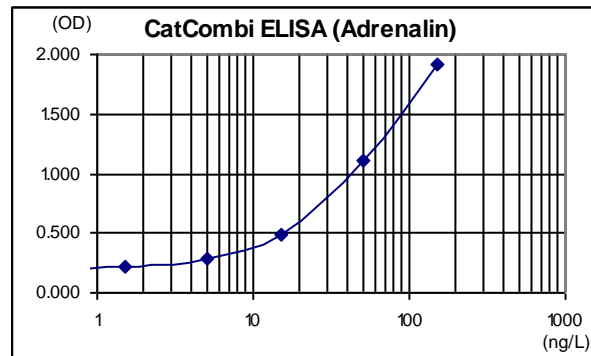

1. Standard	2. Adrenalin (ng/mL)	3. OD _{Mean}	4. OD/OD _{max} (%)
5. A	6. 0.0	7. 0.183	8. 0.0
9. B	10. 1.5	11. 0.215	12. 1.8
13. C	14. 5.0	15. 0.291	16. 6.2
17. D	18. 15	19. 0.490	20. 17.7
21. E	22. 50	23. 1.104	24. 53.2
25. F	26. 150	27. 1.914	28. 100

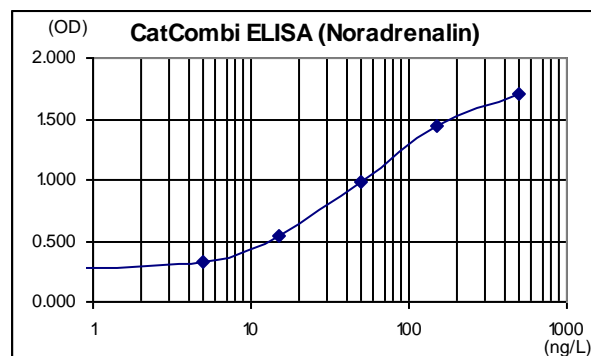



REVISED 7 SEPT. 2011 RM (VERS. 5.1)

RUO IN THE USA

29. Standard	30. Noradrenalin (ng/mL)	31. OD _{Mean}	32. OD/OD _{max} 33. (%)
34. A	35. 0.0	36. 0.223	37. 0.0
38. B	39. 5.0	40. 0.322	41. 6.7
42. C	43. 15	44. 0.539	45. 21.3
46. D	47. 50	48. 0.984	49. 51.2
50. E	51. 150	52. 1.438	53. 81.8
54. F	55. 500	56. 1.708	57. 100

LIMITATIONS OF THE PROCEDURE

Specimen collection has a significant effect on the test results. See SPECIMEN COLLECTION AND STORAGE for details.

For cross-reactivities, see PERFORMANCE.

The following blood components do not have a significant effect (+/-20% of expected) on the test results up to the below stated concentrations:

Hemoglobin	2.0 mg/mL
Bilirubin	1.0 mg/mL
Triglyceride	91 mg/mL

DRG



DRG® CatCombi ELISA (EIA-3175)



REVISED 7 SEPT. 2011 RM (VERS. 5.1)

RUO IN THE USA

PRODUCT LITERATURE REFERENCES

1. Rust MB, Faulhaber J et. al. Neurogenic Mechanisms Contribute to Hypertension in Mice with Disruption of the K-CL Cotransporter KCC3. Circulation Research, January (2006)
2. Creces J., Appleton Ch.: Catecholamines and their Metabolites: Evaluation of a commercial ELISA. Clin. Biochem., QML Pathology, Brisbane QLD (2004)
3. Adams, J. M. et al. Effects of 17 β -Estradiol on hypoglycemia-induced increases in plasma catecholamines in the rat. Poster Society for Neuroscience, Annual Meeting, New Orleans (2003)
4. Westermann J, Hubl W, Kaiser N, Salewski L, Simple, rapid and sensitive determination of epinephrine and norepinephrine in urine and plasma by non-competitive enzyme immunoassay, compared with HPLC method. Clin. Lab., 48: 61-71 (2002)

Version 2011-08-22~rm