



Revised 2 Aug. 2011 rm (Vers. 3.1)

RUO in the USA*This kit is intended for Research Use Only.**Not for use in diagnostic procedures.**Please use only the valid version of the package insert provided with the kit.***NAME AND INTENDED USE**

Anti-TG is an indirect solid phase enzyme immunoassay (ELISA) for measurement of IgG class autoantibodies against thyroglobulin (TG) in human serum or plasma.

PRINCIPLE OF THE TEST

Highly purified human thyroglobulin (TG) is bound to microwells. Antibodies against this antigen, if present in diluted serum or plasma, bind to the respective antigen. Washing of the microwells removes unbound unspecific serum and plasma components. Horseradish peroxidase (HRP) conjugated anti-human IgG immunologically detects the bound antibodies forming a conjugate/antibody/antigen complex. Washing of the microwells removes unbound conjugate. An enzyme substrate in the presence of bound conjugate hydrolyzes to form a blue colour. The addition of an acid stops the reaction forming a yellow end-product. The intensity of this yellow colour is measured photometrically at 450 nm. The amount of colour is directly proportional to the concentration of IgG antibodies present in the original sample.

WARNINGS AND PRECAUTIONS

1. Do not interchange kit components from different lots.
2. Components containing human serum were tested and found negative for HBsAg, HCV, HIV1 and HIV2 by FDA approved methods. No test can guarantee the absence of HBsAg, HCV, HIV1 or HIV2, and so all human serum based reagents in this kit must be handled as though capable of transmitting infection.
3. Avoid contact with the TMB (3,3',5,5'-Tetramethyl-benzidine). If TMB comes into contact with skin, wash thoroughly with water and soap.
4. Avoid contact with the Stop Solution which is acid. If it comes into contact with skin, wash thoroughly with water and seek medical attention.
5. Some kit components (i.e. Controls, Sample buffer and Buffered Wash Solution) contain Sodium Azide as preservative. Sodium azide (NaN_3) is highly toxic and reactive in pure form. At the product concentrations (0.09%), though not hazardous. Despite the classification as non-hazardous, we strongly recommend using prudent laboratory practices (see 8., 9., 10.).
6. Some kit components contain Proclin 300 as preservative. When disposing reagents containing Proclin 300, flush drains with copious amounts of water to dilute the components below active levels.
7. Wear disposable gloves while handling specimens or kit reagents and wash hands thoroughly afterwards.
8. Do not pipette by mouth.
9. Do not eat, drink, smoke or apply makeup in areas where specimens or kit reagents are handled.



Revised 2 Aug. 2011 rm (Vers. 3.1)

RUO

 in the USA

10. Avoid contact between the buffered peroxide solution and easily oxidized materials; extreme temperature may initiate spontaneous combustion.

Observe the guidelines for performing quality control in medical laboratories by assaying controls and/or pooled sera. During handling of all kit reagents, controls and serum samples observe the existing legal regulations.

CONTENTS OF THE KIT

Package size 96 determ.

Qty.1	Divisible microplate consisting of 12 modules of 8 wells each, coated with highly purified human thyroglobulin (TG). Ready to use.
6 vials, 1.5 ml each	Combined Calibrators with IgG class Anti-TG antibodies (A-F) in a serum/buffer matrix (PBS, BSA, NaN ₃ <0,1% (w/w)) containing: IgG: 100; 300; 1000; 3000; 9000 IU/ml Ready to use.
2 vials, 1.5 ml each	Anti-TG Controls in a serum/buffer matrix (PBS, BSA, NaN ₃ <0,1% (w/w)) positive (1) and negative (2), for the respective concentrations see the enclosed QC Certificate. Ready to use.
1 vial, 20 ml	Sample buffer (Tris, NaN ₃ <0.1% (w/w)), yellow, concentrate (5x).
1 vial, 15 ml	Enzyme conjugate solution (PBS, Proclin 300 <0.5% (v/v)), (light red) containing polyclonal anti-human IgG ; labelled with horseradish peroxidase. Ready to use.
1 vial, 15 ml	TMB substrate solution . Ready to use.
1 vial, 15 ml	Stop solution (contains acid). Ready to use.
1 vial, 20 ml	Wash solution (PBS, NaN ₃ <0.1% (w/w)), concentrate (50x).

STORAGE AND STABILITY

1. Store the kit at 2-8 °C.
2. Keep microplate wells sealed in a dry bag with desiccants.
3. The reagents are stable until expiration of the kit.
4. Do not expose test reagents to heat, sun or strong light during storage and usage.
5. Diluted sample buffer and wash buffer are stable for at least 30 days when stored at 2-8 °C.

MATERIALS REQUIRED

Equipment

- Microplate reader capable of endpoint measurements at 450 nm
- Multi-channel dispenser or repeatable pipette for 100 µl
- Vortex mixer
- Pipettes for 10 µl, 100 µl and 1000 µl
- Laboratory timing device
- Data reduction software

**Revised 2 Aug. 2011 rm (Vers. 3.1)****RUO** in the USA**Preparation of reagents**

- Distilled or deionised water
- Graduated cylinder for 100 and 1000 ml
- Plastic container for storage of the wash solution

SPECIMEN COLLECTION, STORAGE AND HANDLING

1. Collect whole blood specimens using acceptable medical techniques to avoid hemolysis.
2. Allow blood to clot and separate the serum by centrifugation.
3. Test serum should be clear and non-hemolysed. Contamination by hemolysis or lipemia is best avoided, but does not interfere with this assay.
4. Specimens may be refrigerated at 2-8 °C for up to five days or stored at -20 °C up to six months.
5. Avoid repetitive freezing and thawing of serum samples. This may result in variable loss of autoantibody activity.
6. Testing of heat-inactivated sera is not recommended.

PROCEDURAL NOTES

1. Do not use kit components beyond their expiration dates.
2. Do not interchange kit components from different lots.
3. All materials must be at room temperature (20-28 °C).
4. Have all reagents and samples ready before start of the assay. Once started, the test must be performed without interruption to get the most reliable and consistent results.
5. Perform the assay steps only in the order indicated.
6. Always use fresh sample dilutions.
7. Pipette all reagents and samples into the bottom of the wells.
8. To avoid carryover contamination, change the tip between samples and different kit controls.
9. It is important to wash microwells thoroughly and remove the last droplets of wash buffer to achieve best results.
10. All incubation steps must be accurately timed.
11. Control sera or pools should routinely be assayed as unknowns to check performance of the reagents and the assay.
12. Do not re-use microplate wells.

For all controls, the respective concentrations are provided on the labels of each vial. Using these concentrations a calibration curve may be calculated to read off the results semi quantitatively.



Revised 2 Aug. 2011 rm (Vers. 3.1)

RUO in the USA**PREPARATION OF REAGENTS*****Preparation of sample buffer***

Dilute the contents of each vial of the sample buffer concentrate (5x) with distilled or deionised water to a final volume of 100 ml prior to use. Store refrigerated: stable at 2-8 °C for at least 30 days after preparation or until the expiration date printed on the label.

Preparation of wash solution

Dilute the contents of each vial of the buffered wash solution concentrate (50x) with distilled or deionised water to a final volume of 1000 ml prior to use. Store refrigerated: stable at 2-8 °C for at least 30 days after preparation or until the expiration date printed on the label.

Sample preparation

Dilute all samples **1:100** with sample buffer before assay.

Therefore combine 10 µl of sample with 990 µl of sample buffer in a polystyrene tube. Mix well.

Controls are ready to use and need not be diluted.

TEST PROCEDURE

1. Prepare a sufficient number of microplate modules to accommodate controls and prediluted samples.
2. Pipette **100 µl** of calibrators, controls and prediluted samples in duplicate into the wells.
3. Incubate for 30 minutes at room temperature (20-28 °C).
4. Discard the contents of the microwells and wash **3 times with 300 µl** of wash solution.
5. Dispense **100 µl** of enzyme conjugate into each well.
6. Incubate for 15 minutes at room temperature.
7. Discard the contents of the microwells and wash **3 times with 300 µl** of wash solution.
8. Dispense **100 µl** of TMB substrate solution into each well.
9. Incubate for 15 minutes at room temperature.
10. Add **100 µl** of stop solution to each well of the modules and incubate for 5 minutes at room temperature.
11. Read the optical density at 450 nm and calculate the results. Bi-chromatic measurement with a reference at 600-690 nm is recommended.

The developed colour is stable for at least 30 minutes. Read optical densities during this time.

Automation

The Anti-Phosphatidyl Serine IgG/IgM ELISA is suitable for use on open automated ELISA processors. The test procedure detailed above is appropriate for use with or without automation.



Revised 2 Aug. 2011 rm (Vers. 3.1)

RUO	in the USA
-----	------------

Calculation of results

For Anti-TG IgG a 4-Parameter-Fit with lin-log coordinates for optical density and concentration is the data reduction method of choice.

Recommended Lin-Log Plot

First calculate the averaged optical densities for each calibrator well. Use lin-log graph paper and plot the averaged optical density of each calibrator versus the concentration. Draw the best fitting curve approximating the path of all calibrator points. The calibrator points may also be connected with straight line segments. The concentration of unknowns may then be estimated from the calibration curve by interpolation.

Calculation example

The figures below show typical test results for the Anti-TG ELISA. These data are intended for illustration only and should not be used to calculate results from another run.

Calibrators									
No	Position	OD 1	OD 2	Mean	Conc. 1	Conc. 2	Mean decl.	Conc.	CV %
S1	A 1/A 2	0.028	0.029	0.028	1.7	1.7	1.7	0.0	0
S2	B 1/B 2	0.184	0.176	0.180	99	93	96	100	4
S3	C 1/C 2	0.452	0.433	0.443	318	300	309	300	4
S4	D 1/D 2	0.970	1.937	0.954	1016	955	986	1000	3
S5	E 1/E 2	1.571	1.555	1.563	3075	2979	3027	3000	1
S6	F 1/F 2	2.011	1.986	1.998	9370	8603	8986	9000	1

REFERENCES

1. Becker,W., Reiners,C., Börner,W. Immunologische Kriterien zur Differentialdiagnose von Schilddrüsenerkrankungen. Therapiewoche 1985;35:1167 - 1176.
2. Czarnocka,B., Ruf,J., Ferrand,M. et al. Purification of the human thyroid peroxidase and its identification as the microsomal antigen involved in autoimmune thyroid diseases.FEBSLett.1985;190:147 - 152.
3. Czarnocka,B., Ruf,J., Ferrand,M. et al. Interaction of highly purified thyroid peroxidase with anti-microsomal antibodies in autoimmune thyroid diseases.J.Endocrinol.Invest.1986;9:135 - 138.
4. Ruf,J., Czarnocka,B., Ferrand,M. et al. Thyroid peroxidase is the organ-specific 'microsomal' antigen involved in thyroid autoimmunity. Acta Endocrinol. (Copenhagen) 1987;Suppl.281:49 -55.
5. Ruf,J., Czarnocka,B., Ferrand,M. et al. Relationship between immunological structure and biochemical properties of human thyroid peroxidase. Endocrinology 1988;Vol.125 (3):1211 - 1218.
6. Horster,F.A. Die Bedeutung von MAK,TAK,TRAK und Thyreoglobulin bei der Diagnose von Schilddrüsenerkrankungen. Internist 1988;29:538 – 540
7. Schatz,H., Löbig,H. Diagnostische und prognostische Bedeutung von Antikörpern gegen Schilddrüsenmikrosomen und Thyreoglobulin. Akt.Endokr.Stoffw.1989;10:146 - 153.
8. Hashimoto,H. Zur Kenntnis der lymphomatösen Veränderung der Schilddrüse (Struma lymphomatosa). Arch.Klin.Chir.1912;97:219 - 248.

**Revised 2 Aug. 2011 rm (Vers. 3.1)****RUO in the USA**

9. Libert,F., Ruel,J., Ludgate,M.et al. Thyroperoxidase, an auto-antigen with a mosaic structure made of nuclear and mitochondrial gene modules. EMBO J.1987; 6:4193 - 4196.
10. McKenzie,J.M., Zakarija,M., Sato,A. Humoral immunity in Graves' disease. Clin. Endocrinol. Metab.1978;7:31 - 45.
11. Scherbaum,W.A., Berg,P.A. Bedeutung von Autoantikörpern in der Diagnostik endokrinologischer Erkrankungen. Dtsch.Med.Wschr.1981;106:308 - 313.
12. Scherbaum,W.A., Berg,P.A. Autoimmune Schilddrüsenerkrankungen - Neuere Aspekte zur Pathogenese und Diagnostik. Dtsch.Med.Wschr.1984;109:1574 - 1581.

Version 2011-06-17~rm