## CLEANER FOR MYTHIC 22

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### INTENDED USE

CLEANER FOR MYTHIC 22 is intended to remove biological material's residues from the measurement system of the hematology analyzers: MTHIC 22, MYTHIC 22AL, MYTHIC 60 and MYTHIC 70. It is only for *in vitro* diagnostics, for healthcare professional users.

### OPERATION PRINCIPLE

CLEANER FOR MYTHIC 22 is a washing reagent located on board of the analyzer. Removal of the biological material residues from the measurement system of the analyzer takes place due to the cleaning properties of the reagent. CLEANER FOR MYTHIC 22 contains surfactants and an enzyme that dissolve and remove large organic particles.

### CONCENTRATION OF THE ACTIVE INGREDIENTS IN THE REAGENT

detergents	0.17%
savinase	0.23%
buffer	
sodium chloride	
preservatives	
stabilizer	

### REAGENT STABILITY

The reagent must be used before the expiry date indicated on the label.

Once opened the reagent is stable for 60 days at room temperature.

### WARNING AND NOTES

- Do not use reagent beyond expiry date printed on the label.
- Do not combine the remaining reagent with the reagent from a new package.
- Use personal protective equipment to prevent contact with samples, reagents and controls.
- Avoid discharge to sewage system or to environment.
- Please refer to the MSDS for detailed information concerning safe storage and use of the product.

### DEVICES INTENDED TO BE USED IN COMBINATION WITH THE REAGENT

CLEANER FOR MYTHIC 22 reagent is intended to be used in combination with the lysing reagent, diluent and appropriate hematology analyzer.

Name	Cat. No
DILUENT FOR MYTHIC 22	HM22-003-10
OnlyOne	HM22-002-1
MYTHIC 22	N/A
MYTHIC 22AL	N/A
MYTHIC 60	N/A
MYTHIC 70	N/A

### ASSAY PROCEDURE

CLEANER FOR MYTHIC 22 is ready for use.

Follow the analyzer's user manual.

Prior the first use, unopened reagent must be stored for minimum 24 hours at room temperature.

### WASTE MANAGEMENT<sup>1</sup>

After use, the reagents should be handled as potentially infectious and disposed of in accordance with local legal requirements.

- Reagents residues: 18 01 07
- Empty packages: 15 01 02
- Wastewater from the analyzer: 18 01 03\*

### INCIDENTS<sup>2</sup>

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer (website address: incidents@cormay.pl) and the competent authority of the Member State in which the user and/or the patient is established.

**Serious incident** means any incident that directly or indirectly led, might have led or might lead to any of the following:

- the death of a patient, user or other person,
- the temporary or permanent serious deterioration of a patient's, user's or other person's state of health,
- a serious public health threat.

### LITERATURE

- European Commission notice on technical guidance on the classification of waste (2018/C 124/01) of 9 April 2018.
- Regulation (EU) 2017/746 of the European Parliament and of the Council of 5 April 2017 on *in vitro* diagnostic medical devices.

### LIST OF CHANGES

Previous version: 01	Current version: 02
Sections updated: INTENDED USE; DEVICES INTENDED TO BE USED IN COMBINATION WITH THE REAGENT; SYMBOL EXPLANATION	
Information added: AUTHORIZED REPRESENTATIVE	

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**PZ CORMAY S.A.**