

ALL TEST™ Toxo IgG/IgM Combo Rapid Test Cassette (Whole Blood/Serum/Plasma)

Package Insert

REF ITGM-425 English

A rapid test for the qualitative detection of IgG and IgM antibodies to *Toxoplasma Gondii* (*T.gondii*) in human whole blood, serum or plasma.
For professional in vitro diagnostic use only

【INTENDED USE】

The Toxo IgG/IgM Rapid Test Cassette (Whole Blood/Serum/Plasma) is a lateral flow chromatographic immunoassay for the simultaneous detection and differentiation of IgM anti-Toxoplasma Gondii (*T. gondii*) and IgG anti-*T. gondii* in human whole blood, serum or plasma. This kit is intended to be used as two screening tests and as an aid in the diagnosis of infection with *T. gondii*. Any reactive specimen with the Toxo IgG/IgM Combo Rapid Test Cassette must be confirmed with alternative testing method(s) and clinical findings.

【SUMMARY】

T. gondii is an obligate intracellular protozoan parasite with a worldwide distribution^{1,2}. Serological data indicates that approximately 30% of the population of most industrialized nations is chronically infected with the organism³. A variety of serologic tests for antibodies to *T. gondii* have been used as an aid in diagnosis of acute infection and to assess previous exposure to the organism. These tests are the Sabin-Feldman dye test, direct agglutination, indirect hemagglutination, latex agglutination, indirect immunofluorescence, and ELISA^{4,7}. Recently, lateral flow chromatographic immunoassay, such as The Toxo IgG/IgM Combo Rapid Test Cassette was introduced into the clinic for the serodiagnosis of *T. gondii* infection.

【PRINCIPLE】

The Toxo IgG/IgM Combo Rapid Test Cassette (Whole Blood/Serum/Plasma) is a qualitative, lateral flow immunoassay for the detection of IgG and IgM antibodies to *Toxoplasma* in whole blood, serum or plasma specimens. In this test, mouse anti-human IgG and mouse anti-human IgM are coated in the test line regions of the test. During testing, the specimen reacts with *T.gondii* antigen coated particles in the test strip. The mixture then migrates forward on the membrane by capillary action and reacts with the mouse anti-human IgG and mouse anti-human IgM on the membrane in the test line region respectively. The presence of a colored line in the test line region indicates a positive result for *T.gondii* infection, while its absence indicates a negative result for that infection.

To serve as a procedural control, a colored line will always appear in the respective control line regions of all the two strips indicating that proper volume of specimen has been added and membrane wicking has occurred.

【REAGENTS】

The test contains mouse anti-human IgM, mouse anti-human IgG and *Toxoplasma T.gondii* antigen. A goat anti-mouse IgG is employed in the control line system.

【PRECAUTIONS】

- For in vitro diagnostic use only. Do not use after the expiration date.
- Do not smoke, drink, or eat in areas where specimens or kits reagents are handled.
- Dispose of all specimens and materials used to perform the test as biohazardous waste.
- This package insert must be read completely before performing the test.
- Bring all reagents to room temperature (15-30°C) before use.

【STORAGE AND STABILITY】

Store as packaged in the sealed pouch at room temperature or refrigerated (2-30°C). The test is stable through the expiration date printed on the sealed pouch. The test must remain in the sealed pouch until use. **DO NOT FREEZE**. Do not use beyond the expiration date.

【SPECIMEN COLLECTION AND PREPARATION】

- The Toxo IgG/IgM Combo Rapid Test Cassette (Whole Blood/Serum/Plasma) can be performed using whole blood.
- Both Fingerstick Whole Blood and Venipuncture Whole Blood can be used.
- To collect **Fingerstick Whole Blood specimens**:
 - Wash the patient's hand with soap and warm water or clean with an alcohol swab. Allow to dry.
 - Massage the hand without touching the puncture site by rubbing down the hand towards the fingertip of the middle or ring finger.
 - Puncture the skin with a sterile lancet. Wipe away the first sign of blood.
 - Gently rub the hand from wrist to palm to finger to form a rounded drop of blood over the puncture site.
- Testing should be performed immediately after specimen collection. Do not leave the specimens at room temperature for prolonged periods. Whole blood collected by venipuncture should be stored at 2-8°C if the test is to be run within 2 days of collection. For long term storage, specimens should be kept below -20°C. Whole blood collected by fingerstick should be tested immediately.
- Bring specimens to room temperature prior to testing. Frozen specimens must be completely thawed and mixed well prior to testing. Specimens should not be frozen and thawed repeatedly for more than three times.
- If specimens are to be shipped, they should be packed in compliance with federal regulations covering the transportation of etiologic agents.

【MATERIALS】

Materials provided

- Test cassettes
- Droppers
- Package insert
- Buffer


Materials required but not provided

- Specimen collection container
- Centrifuge (For plasma only)
- Timer

【DIRECTIONS FOR USE】

Allow the test, specimen, buffer and/or controls to reach room temperature (15-30°C) prior to testing.

- Remove the test cassette from the sealed pouch and use it within one hour. Best results will be obtained if the assay is performed as soon as possible.
- Place the test cassette on a clean and level surface. Hold the dropper vertically; draw the specimen about **1cm above** the upper end of the nozzle as shown in illustration below. Transfer 1 full drop (**approx. 20µL**) of specimen to each sample well, then add 2 drops of buffer (approximately 80µL) to each sample well and start the timer. See the illustration below.
- Wait for the colored line(s) to appear. The result should be read at 15 minutes. Do not interpret results after 20 minutes.

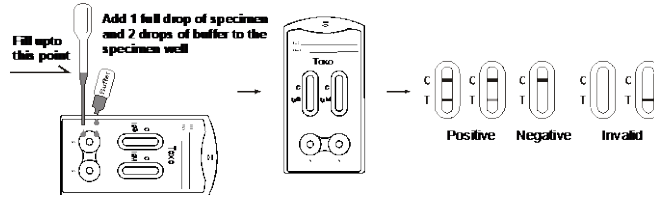

【INTERPRETATION OF RESULTS】

(Please refer to the illustration above)

POSITIVE: Two colored lines appear. One colored line should always appear in the control line region (C) and another line should be in the test line region.

***NOTE:** The intensity of the color in the test line regions may vary depending on the concentration of *T.gondii* IgG or IgM antibodies present in the specimen. Therefore, any shade of color in the test line region should be considered positive.

NEGATIVE: One colored line appears in the control line region (C). No line appears in the test line regions.

INVALID: Control line fails to appear. Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for control line failure. Review the procedure and repeat the test with a new test. If the problem persists, discontinue using the test kit immediately and contact your local distributor.

【QUALITY CONTROL】

Internal procedural controls are included in the test. A colored line appearing in the control region (C) is an internal positive procedural control. It confirms sufficient specimen volume and correct procedural technique.

【LIMITATIONS】

- The Assay Procedure and the Test Result Interpretation must be followed closely when testing the presence of antibodies to *T.gondii* in whole blood, serum or plasma from individual subjects. Failure to follow the procedure may give inaccurate results.
- The Toxo IgG/IgM Combo Rapid Test Cassette is limited to the qualitative detection of the antibodies to *T.gondii* in human whole blood, serum or plasma. The intensity of the test band does not linearly correlate with the antibody titer in the specimen.
- A negative result for an individual subject indicates absence of detectable *T.gondii* antibodies. However, a negative test result does not preclude the possibility of exposure to or infection with *T.gondii*.
- A negative result can occur if the quantity of the *T.gondii* antibodies present in the specimen is below the detection limits of the assay, or the antibodies that are detected are not present during the stage of disease in which a sample is collected.
- Some specimens containing unusually high titer of heterophile antibodies or rheumatoid factor may affect expected results.
- The results obtained with this test should only be interpreted in conjunction with other diagnostic procedures and clinical findings.

【EXPECTED VALUES】

The Toxo IgG/IgM Combo Rapid Test Cassette (Whole Blood/Serum/Plasma) has been compared with a leading commercial Toxo IgG/IgM ELISA test. The correlation between these two systems is 98.2%.

【PERFORMANCE CHARACTERISTICS】

Sensitivity and Specificity

A clinical evaluation was conducted comparing the results obtained using the Toxo IgG/IgM Combo Rapid Test Cassette to Toxo IgG/IgM ELISA Testing. The study included 450 IgG specimens and 450 IgM specimens, and about the IgM specimen both assays identified 395 negative and 47 positive results, about the IgG specimen both assays identified 394 negative and 48 positive results.

Method	IgM Results			Total Results
	T.Gondii EIA (IgM)			
	Results	Positive	Negative	
Toxo IgM Rapid Test Cassette	Positive	47	5	52
	Negative	3	395	398
Total Results		50	400	450

Relative Sensitivity: 94.0% (95%CI*: 83.5%-98.7%)

Relative Specificity: 98.8% (95%CI*: 97.1%-99.6%)

*Confidence Interval

Accuracy: 98.2% (95%CI*: 96.5%-99.2%)

Method	IgG Results			Total Results
	T.Gondii EIA (IgG)			
	Results	Positive	Negative	
Toxo IgG Rapid Test Cassette	Positive	48	6	54
	Negative	2	394	396
Total Results		50	400	450

Relative Sensitivity: 96.0% (95%CI*: 86.3%-99.5%)

Relative Specificity: 98.5% (95%CI*: 96.8%-99.4%)

Accuracy: 98.2% (95%CI*: 96.5%-99.2%)

*Confidence Interval

Precision

Intra-Assay

Within-run precision has been determined by using 10 replicates of three specimens: a negative, a low positive, and a high positive. The negative, low positive, and high positive values were correctly identified >99% of the time.

Inter-Assay

Between-run precision has been determined by 10 independent assays on the same three specimens: a negative, a low positive, and a high positive. Three different lots of the Toxo IgG/IgM Combo Rapid Test cassette (Whole Blood/Serum/Plasma) have been tested over a 10-days period using negative, low positive, and high positive specimens. The specimens were correctly identified >99% of the time.

Cross-reactivity

The Toxo IgG/IgM Combo Rapid Test Cassette (Whole Blood/Serum/Plasma) has been tested for HBsAg, HBsAb, HbeAg, HBeAb, HcAb, HCV, HIV, Syphilis, H. Pylori, HSV 1/2, CMV and Rubella positive specimens. The results showed no cross-reactivity.

Interfering Substances

The following compounds have also been tested using the Toxo IgG/IgM Combo Rapid Test Cassette (Whole Blood/Serum/Plasma) and no interference was observed.

Acetaminophen: 20mg/dl	Caffeine: 20mg/dl	EDTA: 20mg/dl
Acetylsalicylic Acid: 20mg/dl	Geriatric Acid: 20mg/dl	Ethanol: 10%
Ascorbic Acid: 2g/dl	Phenylpropranolamine: 20mg/dl	Glucose: 20mg/dl
Bilirubin: 1000mg/dl	Salicylic Acid: 20mg/dl	Phenothiazine: 20mg/dl

【BIBLIOGRAPHY】

- Krick JA and Remington JS: Toxoplasmosis in the adult: An overview. *New Eng. J. Med.* 1978, 298:550-553
- Anderson SE and Remington JS: The diagnosis of Toxoplasmosis. *So. Med. J.* 1975, 68:1433-1443
- Wilson CB, Remington JS, Stagno S, and Reynolds DW: Development of adverse sequelae in children born with subclinical congenital *Toxoplasma* infection. *Pediatrics*, 1980, 66:767-774
- Berrebi A; Kobuch WE; Bessieres MH; Bloom MC; Rolland M; Sarraon MF; Roques C; Fournie A: Termination of pregnancy for maternal Toxoplasmosis. *Lancet* 1994, 344:36-9
- Fraser KB, Shirodaira PV, and Starford CF: Fluorescent staining and human IgM. *Br. Med. J.* 1971, 3:707
- Pyndiah N, Krich U, Price P and Wilhelm J: Simplified chromatographic separation of immunoglobulin M from G and its application to *Toxoplasma* indirect immunofluorescence. *J. Clin. Micro.* 1979, 9:170-174
- Montoya JG, Rosso F. Diagnosis and management of Toxoplasmosis. *Clin Perinatol.* 2005, 32(3):705-26.

Index of Symbols

	Attention, see instructions for use		Tests per kit		Do not reuse
	For in vitro diagnostic use only		Use by		Catalog #
	Store between 2-30°C		Lot Number		

Hangzhou AllTest Biotech Co., Ltd.
#250, Yinhai Street
Hangzhou Economic & Technological Development Area
Hangzhou 310018, P. R. China
www.alltests.com.cn

Number: 145404101

Effective date: 2017-02-10