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**«DIAQUICK» FERTILITY TEST**

**for home use**

**easy prediction of your fertile days**

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**MATERIALS PROVIDED**

- 5 test devices
- 5 droppers
- package insert

**MATERIALS REQUIRED BUT NOT PROVIDED**

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**Example:** My usual cycle length is 28 days. My last period started on the 3rd. The "when to start testing chart" shows that I will count ahead 11 days beginning with the 3rd. When I count 11 days ahead on the calendar, I find that I will collect and test my urine starting on the 13th.

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- Urinatc into a clean, dry cup or container.
- Remove the test device from the foil pouch and familiarize yourself with the product.
- Hold the dropper vertically and transfer 3 full drops of urine to the specimen well (S) of the test device, and then begin timing. Avoid trapping air bubbles in the specimen well (S).
- As the test begins to work, you may notice a light colored flow moving across the test region (T) and control region (C) on the test device. Read the result at 5 minutes. If no colored line appears, wait 1 minute longer. Do not read the result after 10 minutes.

	<p><b>POSITIVE</b></p>
	<p><b>NEGATIVE</b></p>
	<p><b>INVALID</b></p>

## «DIAQUICK» FERTILITY TEST PRODUCT INSERT ENGLISH

REF Z05515

**CE** 0408

**READING THE RESULTS**

**POSITIV**

Two colored lines are visible, and the test line (T) is the same as or darker than the control line (C). This indicates that you will probably ovulate in 24-36 hours.

**NEGATIV**

Two colored lines are visible, but the test line (T) is lighter than the control line (C), or there is no test line (T). This indicates that no LH surge has been detected and daily testing should be continued.

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**QUESTIONS & ANSWERS**

**Question:** Can I use the «DIAQUICK» Fertility Test to avoid pregnancy?

**Answer:** No, the test should not be used as a form of birth control.

**Question:** How accurate is the «DIAQUICK» Fertility Test?

**Answer:** A clinical evaluation was conducted comparing the results obtained using the «DIAQUICK» Fertility Test to another commercially available urine membrane LH test. The in-house clinical trial included 1,000 urine specimens. The results demonstrated >99.9% specificity and >99.9% sensitivity with an overall accuracy of >99.9%.

**Question:** How sensitive is the «DIAQUICK» Fertility Test?

**Answer:** The «DIAQUICK» Fertility Test detects luteinizing hormone (LH) in urine at concentrations of 40 mIU/mL or higher. The addition of FSH (1,000 mIU/mL), hCG (10,000 mIU/mL), and TSH (1,000 µIU/mL) to negative (0 mIU/mL LH) and positive (40 mIU/mL LH) specimens showed no cross-reactivity.

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**Antwort:** Nein, but you should consult your physician if you are taking any hormonal medication. Also, recent oral contraceptive use, breastfeeding, or pregnancy could affect the test results.

**Question:** Why should I not use first morning urine? What time of the day should I perform the test?

**Answer:** We do not recommend first morning urine because it is concentrated and may give a false positive result. Any other time of day is suitable. For best results, try to collect your urine at approximately the same time each day.

**Question:** Will the amount of liquid I drink affect the result?

**Answer:** Heavy intake of fluids prior to testing will dilute the hormone in your urine. We suggest that you limit your fluid intake for about two hours before you perform the test.

**Question:** How long will the lines remain visible?

**Answer:** The test should be read at 5 minutes for best results. A positive (Surge) result will never disappear. The colored line(s) may become darker and a tinted background may appear after several hours. Some negative results may later display a faint second color line because of evaporation from the test line. Therefore, you should not read the results after 10 minutes. Discard the test once you have read the result.

**Question:** Once I see a positive result, when is the best time to have intercourse?

**Answer:** Ovulation is likely to occur within 24-36 hours. This is your most fertile time. Sexual intercourse within this time frame is advised.

**Question:** I am now using the basal body temperature method (BBT). Does this test replace BBT?

**Answer:** The «DIAQUICK» Fertility Test does not replace the BBT method. The shift in basal body temperature primarily indicates that ovulation has already occurred. The «DIAQUICK» Fertility Test indicates that ovulation is about to occur.

**Question:** I received a positive result and had intercourse during these fertile days but I have not become pregnant. What shall I do?

**Answer:** There are many factors that can affect your ability to become pregnant. It can take normal, healthy couples many months to achieve a pregnancy and often you may need to use the test kit for 3-4 months before achieving pregnancy. If pregnancy is not achieved after 3-4 months, you and your partner should consult your physician.

**Question:** I have had a positive result and had intercourse during these fertile days. I think I may be pregnant. How soon can I find out?

**Answer:** Modern pregnancy rapid tests can provide you with results as early as the first day of your missed period.

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«DIAQUICK» FERTILITY TEST

for home use

easy prediction of your fertile days

PLEASE NOTE

Carefully read this manual before using this product and follow the instructions.

TESTPRINCIPLE

Ovulation is the release of an egg from the ovary. The egg then passes into the fallopian tube where it is ready to be fertilized. In order for pregnancy to occur, the egg must be fertilized by sperm within 24 hours after its release. Immediately prior to ovulation, the body produces a large amount of luteinizing hormone (LH), which triggers the release of a ripened egg from the ovary. This is known as an "LH surge", and usually takes place in the middle of the menstrual cycle. It is during this fertile time that pregnancy is most likely to occur. It is important to note that an LH surge and ovulation may not occur in all cycles. The «DIAQUICK» Fertility Test is a rapid, one-step lateral flow immunoassay for the qualitative detection of LH surges in urine, signaling that ovulation is likely to occur in the next 24-36 hours. The test utilizes a combination of antibodies including a monoclonal LH antibody to selectively detect elevated levels of LH. The results are obtained from the colored lines.

PRECAUTIONS

**Please read all the information in this package insert before performing the test.**

- Do not use after the expiration date printed on the foil pouch.
- Store in a dry place at 2-30°C or 36-86°F. Do not freeze.
- Do not use if pouch is torn or damaged.
- Keep out of the reach of children.
- For in vitro diagnostic use. Not to be taken internally.
- Do not open the test foil pouch until you are ready to start the test.
- The used test should be discarded according to local regulations.
- Each test is to be used once only.

MATERIALS PROVIDED

- 5 test strips
- package insert

MATERIALS REQUIRED BUT NOT PROVIDED

- Specimen collection container
- Timer

INSTRUCTIONS FOR USE

**WHO TO START TESTING**

This kit contains 5 tests, allowing you to test over several days to find your most fertile time. Calculate when to start testing using the chart below.

- First, determine your menstrual cycle length. Your menstrual cycle length is the number of days from the first day of your period (menstrual bleeding) to the last day before the next period starts.
- Next, determine the days to count ahead after the period to start testing. Find your menstrual cycle length on the first row of the chart below, and read the corresponding days to count ahead in the second row. This is the number of days after your period to begin testing.
- The first day of your menstruation is the day you should start counting ahead.
- See the example and sample calendar below to determine which day you should begin testing. First morning urine should NOT be used when testing for LH. For best results, you should test around the same time each day. You should reduce your liquid intake approximately 2 hours prior to testing.

**Table 1: when to start testing**

Menstrual cycle length	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38
Days to count ahead	6	6	7	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21

**Table 2: sample calendar**

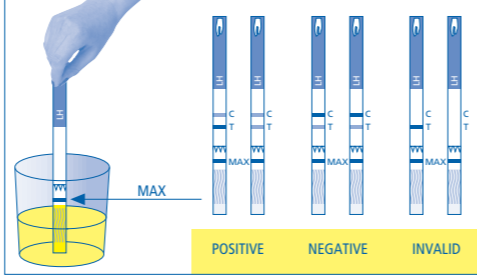
S	M	T	W	T	F	S	<b>Example:</b> My usual cycle length is 28 days. My last period started on the 3rd. The "when to start testing chart" shows that I will count ahead 11 days beginning with the 3rd. When I count 11 days ahead on the calendar, I find that I will collect and test my urine starting on the 13th.
1	2	3	4	5	6		
7	8	9	10	11	12	13	
14	15	16	17	18	19	20	
21	22	23	24	25	26	27	
28	29	30	31				

- first day of the last period
- begin testing with the «DIAQUICK» Fertility Test

**DIRECTIONS FOR USE**

Determine the day you will begin testing using the "When to Start Testing Chart".

- Urinate into a clean, dry cup or container.
- Remove the test strip from the foil pouch and familiarize yourself with the product.
- With arrows pointing toward the urine specimen, immerse the test strip vertically in the urine specimen for at least 10-15 seconds. Do not pass the maximum line (MAX) on the test strip when immersing the strip.
- Place the test strip on a non-absorbent flat surface, start the timer and wait for the red line(s) to appear. As the test begins to work, you may notice a light red flow moving across the test line region (T) and control line region (C) on the test strip. Read the result at 5 minutes. If no red line appears, wait 1 minute longer. Do not read the result after 10 minutes.

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INVALID

Control line fails to appear. The result is invalid if no colored line appears in the control line region (C), even if a line appears in the test line region (T). You should repeat the procedure with a new test.

LIMITATIONS

There is the possibility that this test may produce false positive or false negative results. Invalid results are most likely caused by an insufficient urine specimen or by not following the instructions properly. Review the instructions and repeat the test with a new test. If the problem persists, discontinue using the test kit immediately and contact your local distributor.

QUESTIONS & ANSWERS

**Question:** Can I use the «DIAQUICK» Fertility Test to avoid pregnancy?  
**Answer:** No, the test should not be used as a form of birth control.

**Question:** How accurate is the «DIAQUICK» Fertility Test?  
**Answer:** A clinical evaluation was conducted comparing the results obtained using the Easy Prediction One Step Ovulation Test to another commercially available urine membrane LH test. The in-house clinical trial included 1,000 urine specimens. The results demonstrated >99.9% specificity and >99.9% sensitivity with an overall accuracy of >99.9%.

**Question:** How sensitive is the «DIAQUICK» Fertility Test?  
**Answer:** The «DIAQUICK» Fertility Test detects luteinizing hormone (LH) in urine at concentrations of 40 mIU/mL or higher. The addition of FSH (1,000 mIU/mL), hCG (10,000 mIU/mL), and TSH (1,000 µIU/mL) to negative (0 mIU/mL LH) and positive (40 mIU/mL LH) specimens showed no cross-reactivity.

**Question:** Do alcohol or common medications affect the test?  
**Answer:** No, but you should consult your physician if you are taking any hormonal medication. Also, recent oral contraceptive use, breastfeeding, or pregnancy could affect the test results.

**Question:** Why should I not use first morning urine? What time of the day should I perform the test?  
**Answer:** We do not recommend first morning urine because it is concentrated and may give a false positive result. Any other time of day is suitable. For best results, try to collect your urine at approximately the same time each day.

**Question:** Will the amount of liquid I drink affect the result?  
**Answer:** Heavy intake of fluids prior to testing will dilute the hormone in your urine. We suggest that you limit your fluid intake for about two hours before you perform the test.

**Question:** How long will the lines remain visible?  
**Answer:** The test should be read at 5 minutes for best results. A positive (Surge) result will never disappear. The colored line(s) may become darker and a tinted background may appear after several hours. Some negative results may later display a faint second color line because of evaporation from the test line. Therefore, you should not read the results after 10 minutes. Discard the test once you have read the result.

**Question:** Once I see a positive result, when is the best time to have intercourse?  
**Answer:** Ovulation is likely to occur within 24-36 hours. This is your most fertile time. Sexual intercourse within this time frame is advised.

**Question:** I am now using the basal body temperature method (BBT). Does this test replace BBT?  
**Answer:** The «DIAQUICK» Fertility Test does not replace the BBT method. The shift in basal body temperature primarily indicates that ovulation has already occurred. The «DIAQUICK» Fertility Test indicates that ovulation is about to occur.

**Question:** I received a positive result and had intercourse during these fertile days but I have not become pregnant. What shall I do?  
**Answer:** There are many factors that can affect your ability to become pregnant. It can take normal, healthy couples many months to achieve a pregnancy and often you may need to use the test kit for 3-4 months before achieving pregnancy. If pregnancy is not achieved after 3-4 months, you and your partner should consult your physician.

**Question:** I have had a positive result and had intercourse during these fertile days. I think I may be pregnant. How soon can I find out?  
**Answer:** Modern pregnancy rapid tests can provide you with results as early as the first day of your missed period.

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