



Coag Cleaner WASHING SOLUTION

Cat. No.: h60102

1 x 4,5 L

PRODUCT NAME

Coag Cleaner washing solution.

INTENDED USE

(For In Vitro Diagnostic Use Only)

Coag Cleaner solution is the washing solution of DIAGON Coag Line automated coagulation analysers for diagnostic measurements.

SUMMARY AND EXPLANATIONS

Coag Cleaner solution is a surfactant containing washing solution, that serves for daily maintenance and cleaning of coagulation analysers.

ACTIVE INGREDIENTS

Coag Cleaner solution contains surfactant (<1,0%) and preservative (<0,5%).

PRECAUTIONS

- Person installing the Coag Cleaner solution must be a trained laboratory professional!
- Solution coming into contact with specimens and other materials should be handled as if capable of transmitting infection and should be disposed of with proper precautions!
- Avoid microbial contamination of the solution otherwise erroneous results may occur!
- All solutions, waste and utilized disposable laboratory equipment should be considered as hazardous waste! Their handling and disposal should be done according to the valid hazardous material processing regulation.
- Do not use the solution beyond the expiration date printed on the label!

PREPARATION

Coag Cleaner solution is ready to use. Wait until the solution reaches the working temperature!

TEST PROCEDURE

For proper application of Coag Cleaner solution, please follow the instructions of the instrument's manual.

STORAGE AND STABILITY

Coag Cleaner solution in intact cubitainer is stable until the expiration date given on the label, when stored at 15-35°C. Stability after opening in the original cubitainer is shown in below table:

| T (°C) | 15-35 |
|--------|-------|
| Day | 60 |

EXPECTED RESULTS

Coag Cleaner solution will effectively clean the tubes in the instrument used on daily basis and keeping carryover values within specification.

MATERIALS REQUIRED BUT NOT PROVIDED

Coagulation reagents

Tel.:

• DIAGON Coag Line automated coagulation analyser for measuring.

DIAGON LTD.



Baross u. 48-52, 1047 Budapest, Hungary +36 1 3696500 +36 1 3696301 www.diagon.com E-mail: diagon@diagon.com

| SYMBOLS | | | | |
|---------|--------------------------|-------------|------------------------------------|--|
| | Manufacturer | $\geq \leq$ | Use-by date | |
| LOT | Batch code | REF | Catalogue number | |
| Ť | Keep dry | 35 °C | Temperature limit | |
| Ŵ | Handle with care | Ĩ | Consult instruction for use | |
| [F] | Do not use hand hooks | IVD | In vitro diagnostic medical device | |
| Œ | CE mark | <u>††</u> | This side up | |