CORMAY D-DIMER CONTROLS

CONTROL SERUM (LYOPHILISED) FOR DETERMINATION OF D-DIMER CONCENTRATION

CORMAY D-DIMER CONTROLS 4 x 0.5 ml Cat. No 4-459

INTENDED USE

CORMAY D-DIMER CONTROLS is designed for use as control serum when measuring D-dimer concentration. Kit contains lyophilised control sera containing D-dimer at two levels (low – L and high – H). The exact concentration of D-dimer is printed on the label.

RECONSTITUTION

Carefully remove the rubber cap from vial and add 0.5 ml D-DIMER DILUENT to the vial. Recap and leave at room temperature for 15 minutes, inverting gently to dissolve contents.

STABILITY AND STORAGE

1. Lyophilised serum remains stable when stored at 2-10°C until expiry date given on the product label.
2. The constituents in reconstituted control serum are stable for 2 weeks at 2-10°C.

NOTES

1. Product for in vitro diagnostic use only.
2. Control serum must be used only for the purpose intended by suitably qualified laboratory personnel, under appropriate laboratory conditions.
3. This control serum has been tested for the HIV antibody, HBsAg and HCV and found to be non-reactive. However this material should be handled as though capable of transmitting infectious disease.
4. D-DIMER DILUENT is present in kits for determination of D-dimer concentration.
5. D-DIMER DILUENT contains sodium azide (< 1 g/l) as a preservative. Avoid contact with skin and mucous membranes.

WASTE MANAGEMENT

Please refer to local legal requirements.

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